Wi-Fi Key

- de** Vor der Installation beachten
- en** Notes before installation
- es** Tener en cuenta antes de la instalación
- fr** A respecter avant l'installation

de	4
en	9
es	14
fr	19

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en - Contents

Notes before installation	9
Range and availability of WiFi signal.....	9
Routers.....	9
Routers with access restrictions.....	9
Routers with time restrictions.....	9
Setting up the storage location (remote folder) for the cycle reports.....	10
Activating remote data storage on the CUBE.....	11
Process documentation	12
Miele DataDiary app.....	12
Downloading the app.....	12

Read the operating instructions for the CUBE/CUBE X and the WiFi dongle carefully before using them.

Detailed operating instructions for the CUBE/CUBE X can be found on the CUBE/CUBE X USB storage device or under www.miele.com/professional.

Keep the operating instructions in a safe place.

Range and availability of WiFi signal

The WiFi range or signal strength is dependent on the structural environment. Reinforced concrete walls and ceilings, metal surfaces, metal-vaporised films, glass screens and fire protection constructions can muffle or divert the signal, therefore reducing the range.

For example, if the CUBE/CUBE X is located under a metal worktop, or is surrounded by metal cabinets, the signal will be almost completely blocked.

Routers

Routers with access restrictions

Routers with access restrictions only allow certain devices to access the network, e.g. with the aid of a MAC filter.

- Please contact your system administrator so they can set up the relevant access.

Routers with time restrictions

Routers with time restrictions allow data to be transmitted within a particular time frame.

- If the process ends outside of this time frame, the process data must be downloaded manually on the next working day.

Setting up the storage location (remote folder) for the cycle reports

A fixed storage location (remote folder; folder for file storage) is required for saving the cycle reports. The appropriate write permissions for the folder must be assigned to the CUBE user. Otherwise it will not be possible to transfer any reports. If necessary, a new user must be created with write permissions in your operating system, e.g. the user “CUBE” or “CUBE X”. Please contact your administrator for this purpose.

- Set up a storage location (folder) for the cycle reports in your network or on the storage device.
- Assign write permissions for the folder to the CUBE user.

The folder path must be entered on the display of the CUBE/CUBE X. For information on how to do this, refer to the operating instructions for the CUBE/CUBE X.

Activating remote data storage on the CUBE

Remote data storage must be activated as a one-off process in the CUBE control system when installing the WiFi dongle.

Remote data storage only needs to be activated on the CUBE.

On the CUBE X, data storage is activated ex works.

- Install the WiFi dongle in accordance with the instructions supplied with the dongle.
- Switch the CUBE on.
- Open the main menu in the CUBE display by tapping the  symbol.
- Open the Accessories menu by tapping the  symbol.
- Tap the  symbol to call up the input screen for special codes.
- Enter the code below to activate data storage. Please note that it is case-sensitive:
 - CUBEDataremoteE
- Confirm the entry with .

Data storage is now activated. To set up the remote folder, refer to the CUBE operating instructions.

- Finally, test the data storage in accordance with the information in the CUBE operating instructions.
- To deactivate data storage, enter the following code:
 - CUBEDataremoteD

en - Process documentation

Miele DataDiary app

The Miele DataDiary app is used for process documentation in the context of instrument reprocessing.

Downloading the app

The Miele DataDiary app can be obtained from the following platforms.

The app is free to download and use.

- Scan the QR code for the relevant platform or enter the web address in your browser.

- Search for the Miele DataDiary app on the platform and download it.

If you would like to download the app on your tablet, it must have at least iOS 10[®] or Android 8[™].

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