Miele

DataDiary

da	Kort vejledning App
de	Kurzanleitung App
en	Brief instructions App
es	Guía rápida de la App
fr	Notice Application
it	Istruzioni brevi App
nl	Korte gebruiksaanwijzing app
pl	Skrócona instrukcja obsługi Aplikacja
pt	Guia de iniciação rápida App
sv	Snabbinstruktion App

da 5
de 51
en 97
es143
fr189
it235
nl280
pl326
pt371
sv416

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Notes about these instructions	97
Definition of terms	97
Intended use	98
After installation	99
Starting the app.....	99
Settings	99
Language.....	99
Creating an administrator	100
Settings complete	101
Saving energy	102
Configuring energy-saving mode	102
Using the machine	103
Protecting access to the app	103
Starting the app.....	103
Dashboard.....	104
Menu	105
Opening the menu.....	105
Options.....	105
Cancelling.....	105
Administration	106
Administration menu	106
Opening the menu.....	106
Logging in and out as administrator.....	106
Options.....	107
Language.....	108
Changing the language	108
User	109
Creating users	109
Managing users.....	111
Managing the administrator	113
Machines	116
Compatible machines	116
Optional components.....	116
Connection options	116
Adding a washer-disinfector	117
Adding CUBE / CUBE X.....	122
Machine management.....	124
WiFi configuration.....	127
Routers.....	127
Range and availability of WiFi signal	128
Archiving reports	128
Save location.....	128
Saving reports to SD cards	129
Reports	130
Live report	130

en - Contents

Editing reports	131
Retrieving reports	131
Checking reports	131
Aborting editing	133
Approving reports.....	134
Cancelling reports	136
Viewing reports.....	137
Retrospective output of reports	138
Information	139
Information	139
Problem solving guide	140

Warnings

⚠ Warnings contain information which is important for safety. This alerts you to the potential danger of injury to people or damage to property.

Read these warning notes carefully and observe the procedural instructions and codes of practice they describe.

Notes

Notes provide information of particular importance that must be observed.

Additional information and comments

Additional information and comments are contained in a simple frame.

Operating steps

Operating steps are indicated by a black square bullet point.

Example:

- Select an option and save your choice with *OK*.

Display

Display text can be identified from the special font.

Example:

Save.

Definition of terms

Machine

Washer-disinfectors and sterilisers are generally referred to as “machines” in these operating instructions.

Process

The machine-based reprocessing procedures, such as the programmes of the washer-disinfectors and the sterilisation cycles of the sterilisers, are generally referred to as “processes”.

DataDiary serves to document process data from Miele washer-disinfectors, whereby data transmission is wireless using the XKM communication module.

Furthermore, DataDiary facilitates the documentation of process data from CUBE and CUBE X small sterilisers, whereby data transmission is wireless using the WiFi module USB flash memory.

DataDiary may only be used in conjunction with the Miele washer-disinfectors and small sterilisers specified below.

Washer-disinfectors

- | | |
|--------------|--------------|
| – PG 8581 | – PG 8583 CD |
| – PG 8582 | – PG 8591 |
| – PG 8582 CD | – PG 8592 |
| – PG 8583 | – PG 8593 |

Small sterilisers

- | | |
|---------------------|---------------------|
| – PST 1710 (CUBE) | – PST 2210 (CUBE) |
| – PST 1720 (CUBE X) | – PST 2220 (CUBE X) |

For wireless data transmission, each washer-disinfector must be equipped with a Miele XKM 3100 WCDS communication module and each small steriliser must be equipped with a Miele APST 002 WiFi module USB flash memory.

Report records issued by the above washer-disinfector or small steriliser models are transmitted “as is” to an app installed on a mobile device (tablet PC).

The XKM communication module and the WiFi module USB flash memory can be integrated into a local WiFi network in the same way as a tablet PC.

Alternatively, the XKM communication module provides an access point for one washer-disinfector to which a single tablet PC can connect (1:1 direct connection).

The process is similar for the small steriliser: this provides an access point via the WiFi module USB flash memory to which a single tablet PC can connect (1:1 direct connection).

Process data from one or more washer-disinfectors and the small sterilisers is processed “as is” in the form of a printer report (digital printer).

An authenticated user can approve or cancel a report on the basis of the printer report which is saved to the tablet PC as a PDF file from where it can be printed.

Starting the app

- Start the DataDiary app by tapping the app icon.

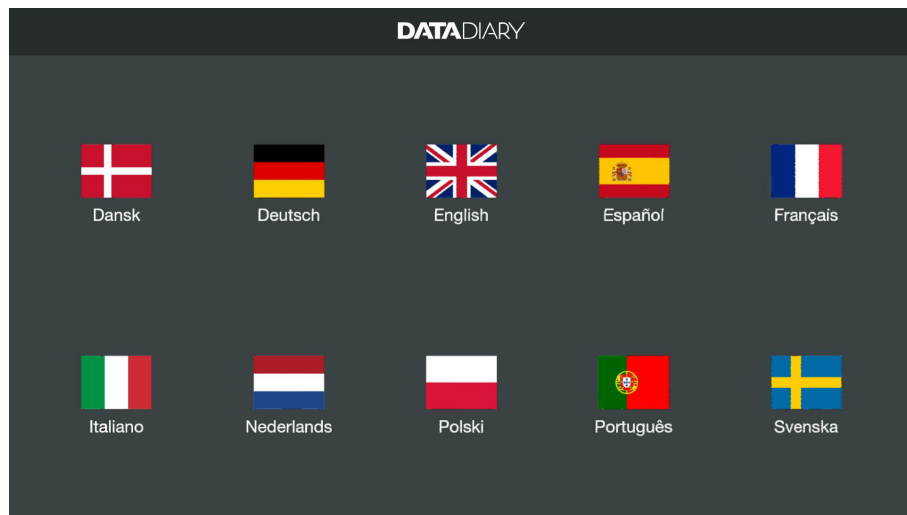
When you start the DataDiary app for the first time following installation, you will need to make a few basic settings.

Settings

Language

First, you will be asked to select the language to be used by the DataDiary app.

- To do this, tap one of the flags.

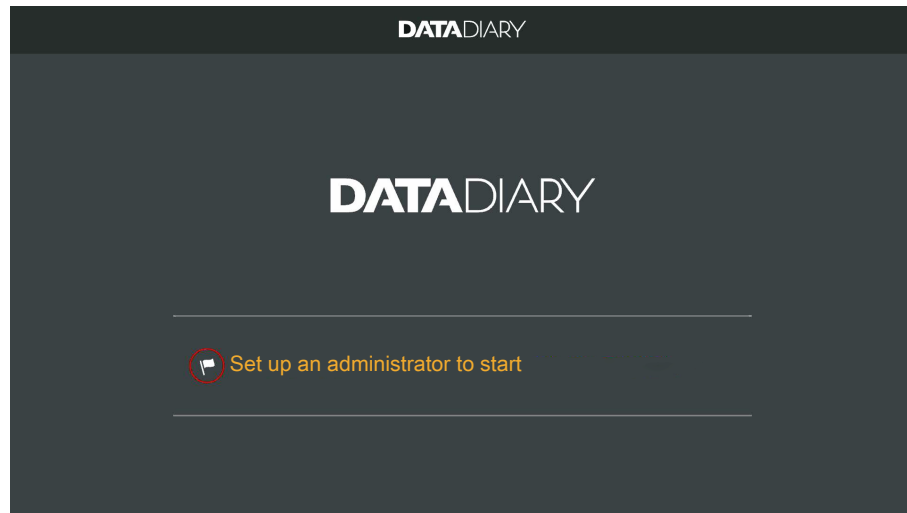
The language you select does not affect the language in which the cycle reports are output.

The report language is set on your cleaning machine or steriliser. To do this, consult the information in the operating instructions for the cleaning machines or sterilisers and the respective WiFi accessories.

en - After installation

Creating an administrator

Next, you will need to create an administrator for the app.

- Tap the text field containing the flag icon .

- Complete the First name, Name and Position fields, all of which are mandatory. When you want to complete a field, start by tapping it.
- Enter a Password and answer the Security question.

Storing an NFC tag You then have the option of storing a chip card with an NFC (near-field communication) tag for the user profile.

If someone logs in using an NFC card, there is no need for them to enter their user name or password.

To use NFC cards, you need a tablet that supports NFC. Suitable NFC cards are supplied with the Miele WiFi accessories. You can obtain additional cards from Miele.

- Hold the NFC card against the back of the tablet. In most cases, the NFC module that reads the NFC card is located at the edge. However, the exact position of the NFC antenna will vary from one tablet model to another.

If the message NFC tag saved xxxxxxxxx appears, where xxxxxxxxx represents the ID number of the NFC card, this means that the card has been successfully assigned to the user profile.

Use **Reset** to remove the NFC card assignment.

Saving

- Tap the **Set up administrator** button.

Settings complete Once you have created the administrator, the basic settings for the app are complete.

You can now log in as the administrator to make further settings (see “Administration”) or you can cancel the login process.

If you choose to cancel, the view switches to the Dashboard of DataDiary. The instructions for using the Dashboard are provided later on in the operating instructions (see “Using the app/Dashboard”).

Configuring energy-saving mode

Most tablets offer various options for saving energy, such as a sleep mode or energy-saving mode. These options reduce the performance of the tablet and restrict the background activities that apps are able to carry out. Depending on the tablet model and the operating system, using these options may abort the transmission of data between the Miele app and the Miele machines.

If you do wish to use energy-saving options on your tablet, please note the following:

- Make sure that the Miele app still remains active in the background if energy-saving mode is used.
- When configuring the WiFi, make sure that you set it to remain active when the tablet is in energy-saving mode.
- Make sure that the tablet battery has enough charge, or use a mains power supply. Some tablet models enter energy-saving mode automatically if their battery level is low. This may impair data transmission or cause it to abort completely.
- If you have set your tablet to shut down automatically at a specific time, make sure you start the process in enough time for it to finish before the tablet shuts down.

For more information about configuring energy-saving mode and WiFi, consult the operating instructions for your tablet.

Protecting access to the app

⚠ Unauthorized access poses a health risk.
Unauthorized access to the app could cause inadequately reprocessed cycles to be approved for further use or settings in the app to be changed, for example.
Do not disclose your password to other app users or third parties. Should your password become known to others, change it and the answer to the security question immediately.
Keep your NFC card safe and do not give it to other users or third parties.
If any NFC cards go missing, delete the assignments immediately.

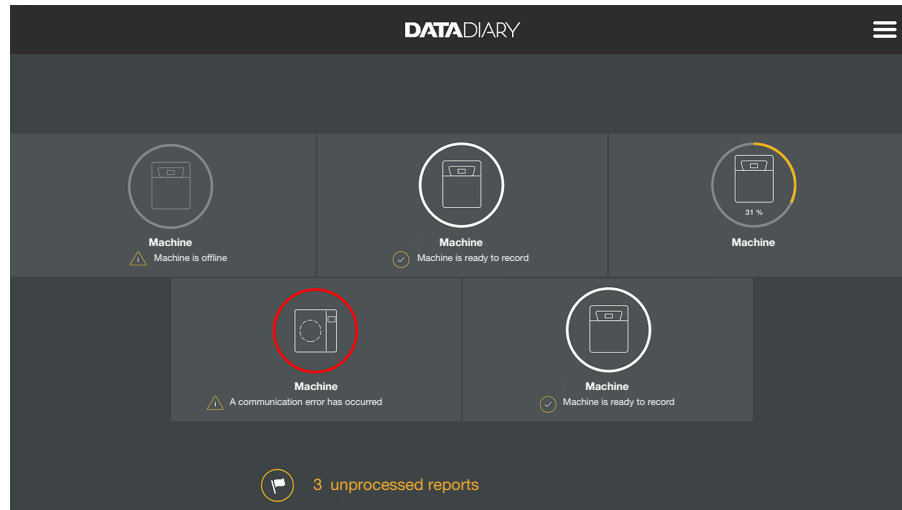
Starting the app

- Start the DataDiary app by tapping the app icon.

The app launches with the Dashboard. The DataDiary app is operated via the Dashboard and the app menu. The following information describes how to open these and the options they contain.

Dashboard

The Dashboard displays all the cleaning machines and sterilisers registered in the app along with their status.

- Grey: machine is offline
- White: machine is online and ready for operation
- Yellow: machine is in operation
- Green (*CUBE/CUBE X only*): a process has been finished successfully (process parameters fulfilled)
- Red: connection error between machine and app

The bottom section of the display indicates any pending tasks, system messages, etc. as necessary.

The machine icons and the pending tasks and system messages are active buttons.

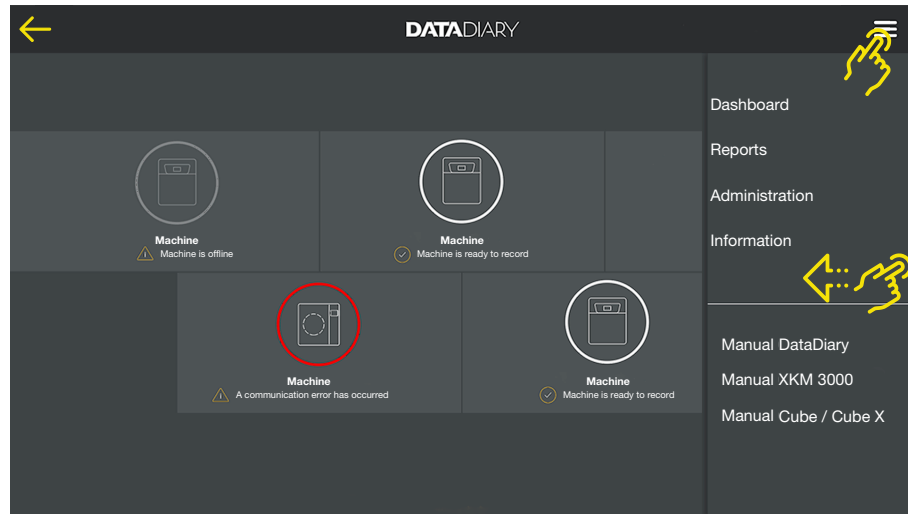
The flag icon  on the Dashboard indicates that there are pending tasks and system messages. The flag icon is an active button.

- Tap one of the buttons to view further details on the status of the machines or to process pending tasks, for example.

Menu

Opening the menu

In addition to the Dashboard, you can also operate the app via the menu. Follow the steps below to access this:

- Tap the  icon at the top right.

Or:

- Swipe from the right-hand edge of the screen towards the centre.

Options

The menu contains the following options:

- Dashboard
Selecting this option takes you back to the Dashboard.
- Reports
Here, you can view and edit stored reports.
- Administration
This menu option allows you to make app settings, such as creating new users, registering new machines or setting up a WiFi connection.
- Information
Here, you can view information about the app, such as the app version or the legal notice.
- The manuals (operating instructions) for the app and the WiFi accessories are also stored here.

Cancelling

Use the arrow icon ← at the top left to exit the menu or return to the next-highest menu level.

Administration menu

Settings such as setting up a WiFi connection, registering new machines or creating additional users are only available to the app administrator and are stored in the Administration menu. The menu is password-protected and can only be opened by the app administrator.

Opening the menu

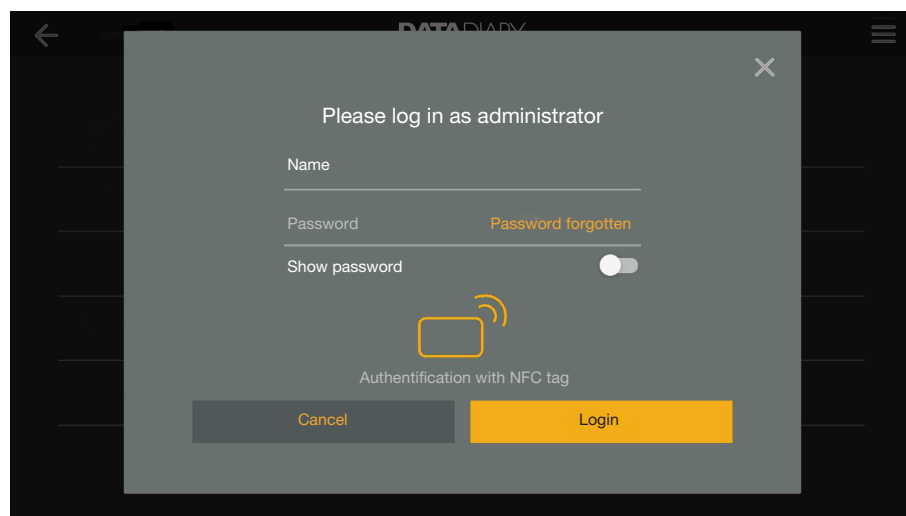
- Open the menu for the DataDiary app and select the Administration option.

You will then be prompted to log in as the administrator.

Logging in and out as administrator

The following instructions describe how to log in and out as the app administrator.

Logging in as administrator

- You can log in as the administrator either by entering your user name and password or by holding the NFC card against the tablet.

Logging out as administrator

- To log back out as administrator, tap Logout at the top right next to the menu  icon.

Automatic logout

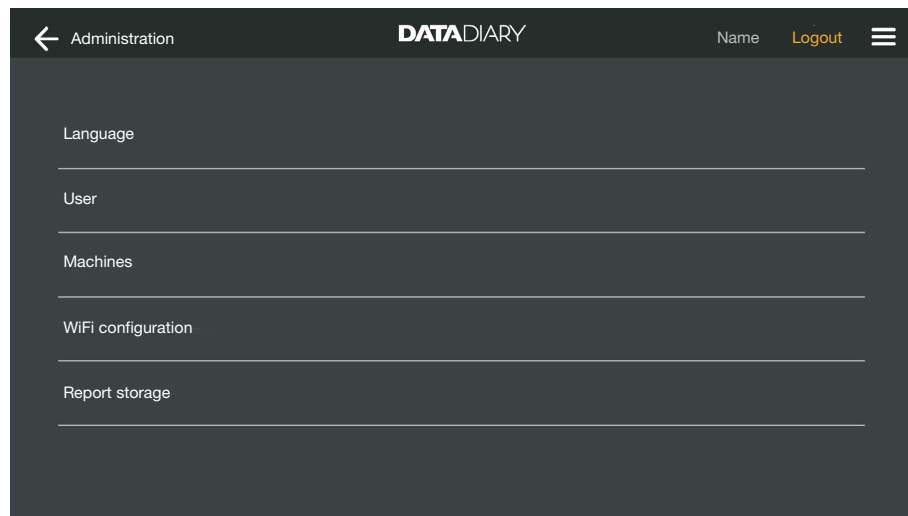
The administrator is automatically logged out under the following conditions:

- If you select the Reports or Information option in the app menu.
- If you select an icon other than  (which is used to register additional machines) on the Dashboard, such as the live report or flag icon . The  icon is only displayed in the administrator view (see “Machines”).

If you have opened the Administration menu, return to the Dashboard from the menu or access the manuals via the app menu, you will stay logged in as the administrator.

Options

Once you have logged in as the administrator, the Administration menu will open.

This contains the following options:

- Language
Setting the app's output language
- User
Creating new users and managing user profiles, including the administrator profile
- Machines
Registering and configuring machines
- WiFi configuration
Configuring the WiFi connection
- Report storage
Selecting the local save location for the cycle reports

The individual options are described below.

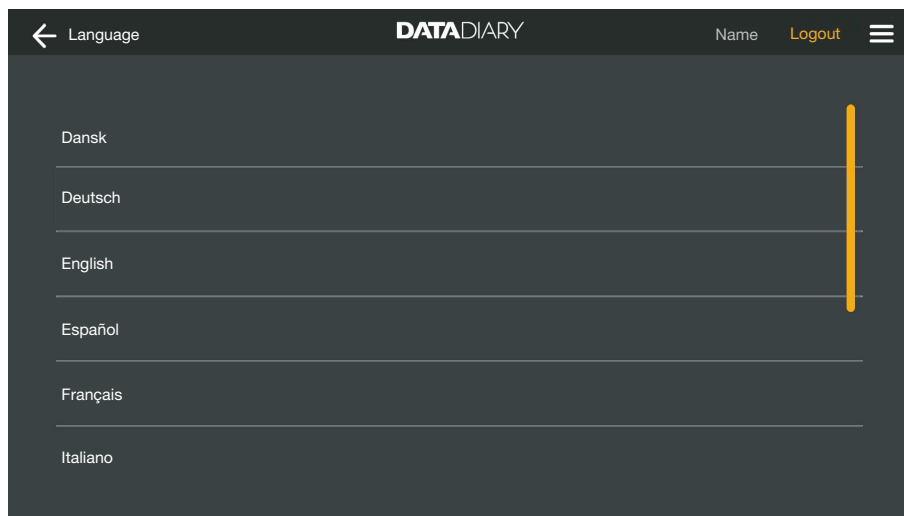
Language

Changing the language

The language you select does not affect the language in which the cycle reports are output.

The report language is set on your cleaning machine or steriliser. To do this, consult the information in the operating instructions for the cleaning machines or sterilisers and the respective WiFi accessories.

- Select the **Administration** option in the menu for the DataDiary app and log in as the app administrator.
- Select the **Language** option.


- Select a language by tapping the relevant field. The language will be changed immediately.
- Exit the menu using the arrow icon ← at the top left.

User

Creating users

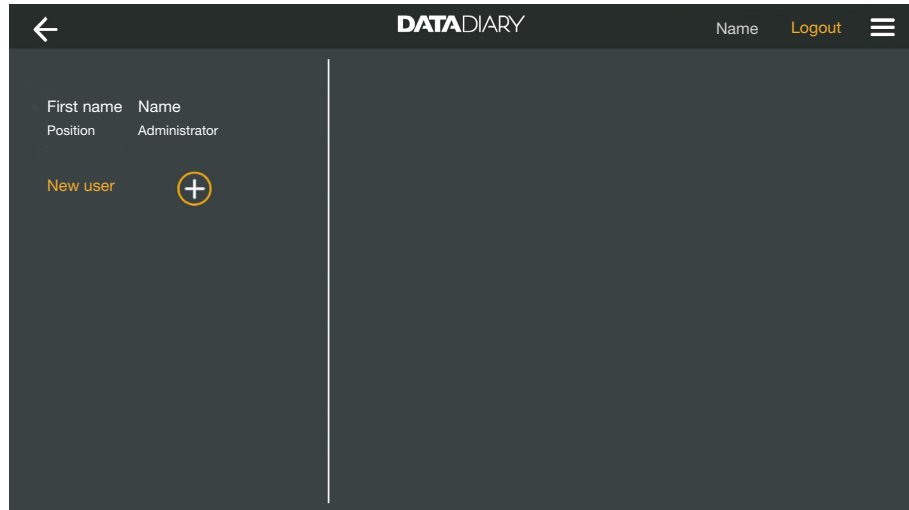
- Select the Administration option in the menu for the DataDiary app and log in as the app administrator.
- Tap the User option.

The screen view splits into two. The left-hand side lists all the saved app users.

- Tap the ⊕ icon to create a new user.

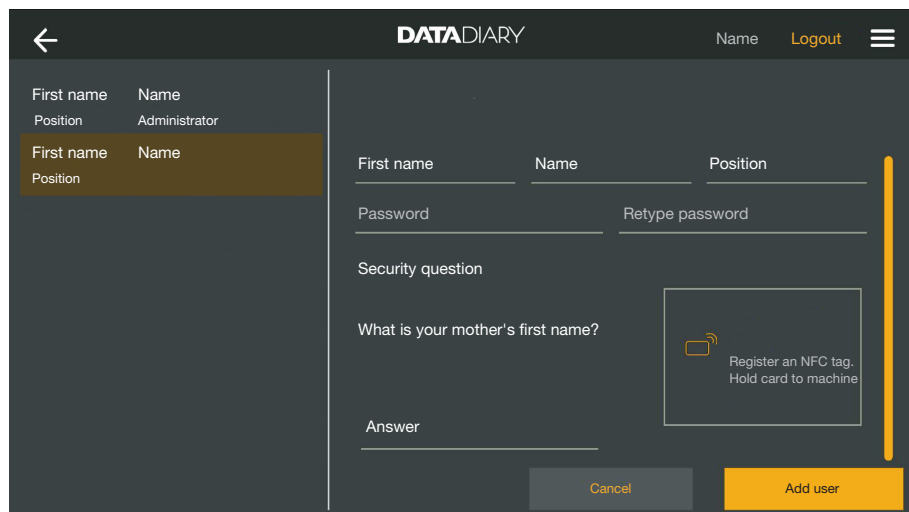If the ⊕ icon is not displayed, the maximum number of users has already been registered with the app. It is not possible to add any more users. Delete individual users as necessary or adapt the existing user profiles.

- Complete the First name, Name and Position fields, all of which are mandatory. When you want to complete a field, start by tapping it.
- If you wish, you can enter a password for the user. Alternatively, you can leave the fields blank so that the new user can create their own password the first time they log in.

en - Administration

- If you wish, you can also answer the security question for the new user. The answer is not mandatory and can only be entered and changed by the administrator.

Storing an NFC tag You then have the option of storing a chip card with an NFC (near-field communication) tag for the user profile.

If someone logs in using an NFC card, there is no need for them to enter their user name or password.

To use NFC cards, you need a tablet that supports NFC. Suitable NFC cards are supplied with the Miele WiFi accessories. You can obtain additional cards from Miele.

- Hold the NFC card against the back of the tablet. In most cases, the NFC module that reads the NFC card is located at the edge. However, the exact position of the NFC antenna will vary from one tablet model to another.

If the message NFC tag saved xxxxxxxxxx appears, where xxxxxxxxxx represents the ID number of the NFC card, this means that the card has been successfully assigned to the user profile.

Use **Reset** to remove the NFC card assignment.

Saving

- Tap the **Add user** button.

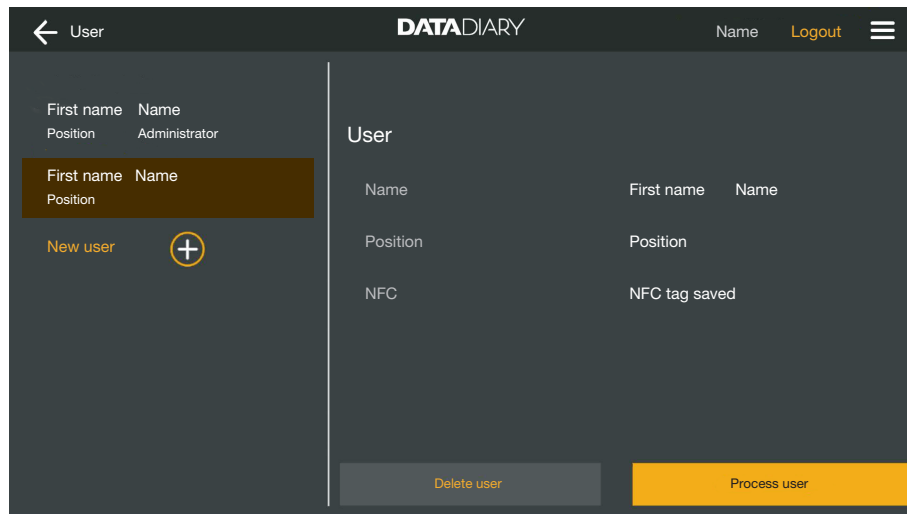
Managing users

- Select the Administration option in the menu for the DataDiary app and log in as the app administrator.
- Tap the User option.

The screen view splits into two. The left-hand side lists all the saved app users.


- Tap one of the users that has already been created, but not the administrator.

The right-hand side of the screen displays the data that is saved for the user. You can now edit or delete the user profile.

Editing users

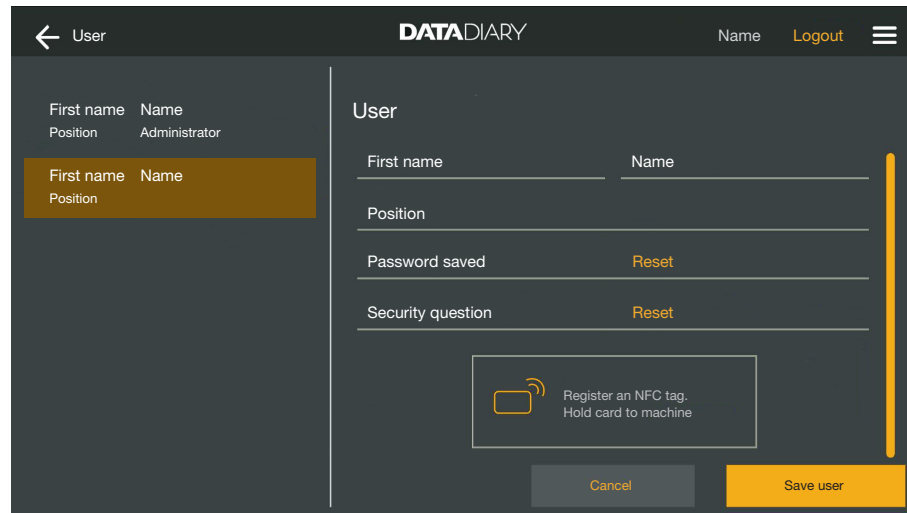- Tap Process user.

You can make the following settings:

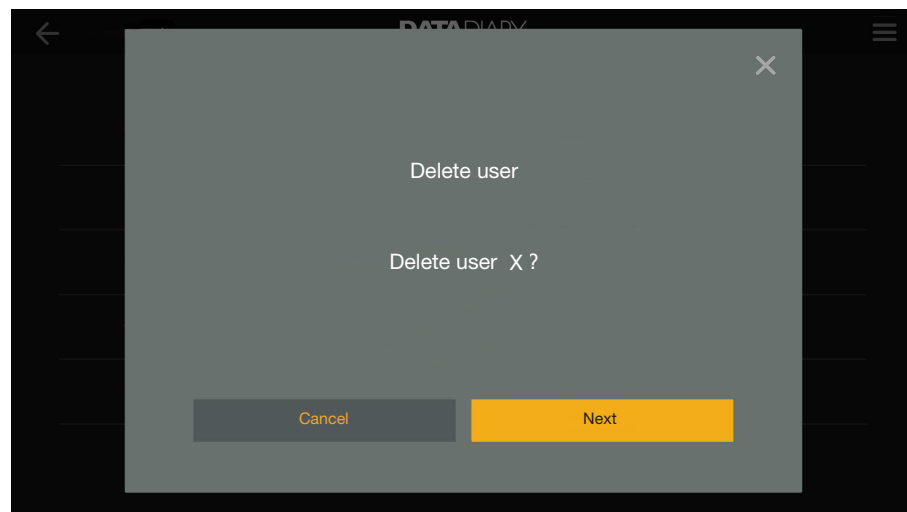
- You can change the name and the position of the user. When you want to complete a field, start by tapping it.
- You can reset the user's password and security question. Tap Reset.

The user can then enter a new password and answer to the security question the next time they log in.

- You can store an NFC card for the user, delete an existing assignment (Reset) or store a new card. For the last option, first tap Reset and then hold the new card against the tablet.
- Tap Save user to save the changes or Cancel to exit without saving.

Deleting users

- Tap Delete user.


A confirmation prompt will appear.

- Tap Next to irretrievably delete the user or Cancel to exit without deleting the user.

Managing the administrator

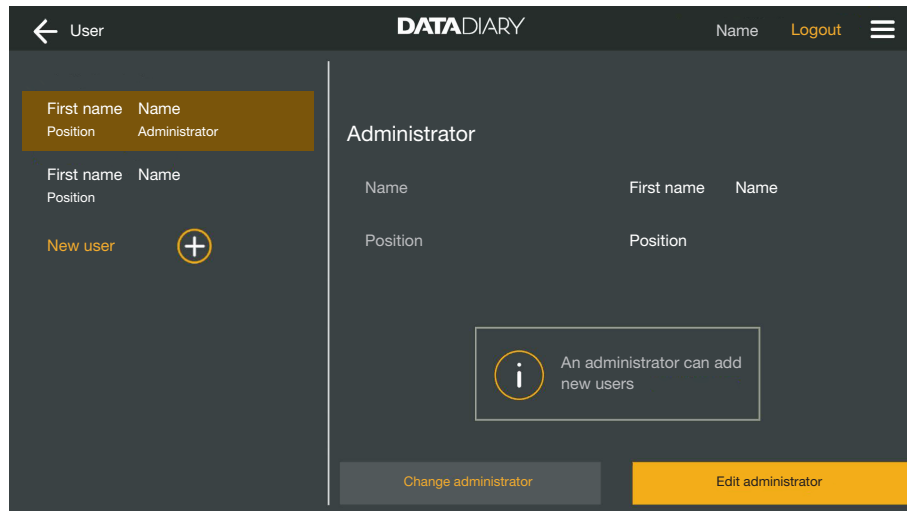
- Select the Administration option in the menu for the DataDiary app and log in as the app administrator.
- Tap the administrator.

The right-hand side of the screen displays the data that is saved for the administrator. You can edit the profile or transfer the administrator rights to a different user profile.

Editing the administrator

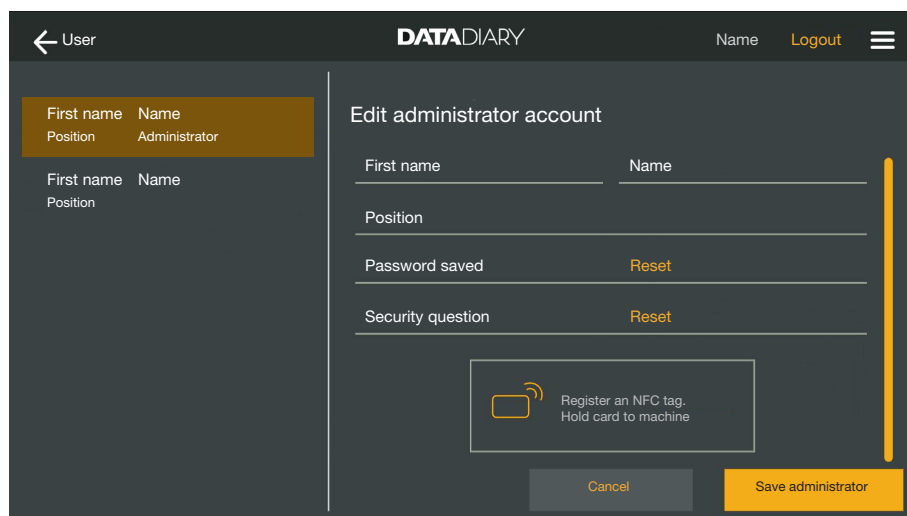- Tap Edit administrator.

You can make the following settings:

- You can change the name and the position. When you want to complete a field, start by tapping it.
- You can reset the password and security question and enter new ones. Tap Reset.
- You can store an NFC card for the administrator, delete an existing assignment (Reset) or store a new card. For the last option, first tap Reset and then hold the new card against the tablet.
- Tap Save administrator to save the changes or Cancel to exit without saving.

en - Administration

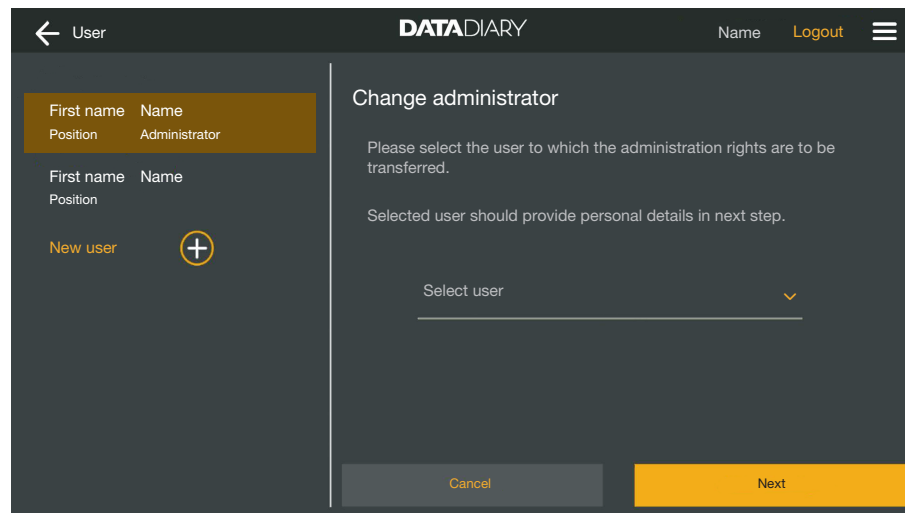
Changing the administrator

You can transfer the administrator rights to another user. At least 1 other user profile must have been created to do this.

Only 1 administrator profile can be created for the DataDiary app. It is not possible to create multiple parallel administrators for the app.


If you wish to transfer the administrator rights to another user, make sure that this user is present when you transfer the rights. The new administrator has to enter certain details themselves.

- Tap Change administrator.

The screenshot shows the 'Change administrator' screen in the DataDiary app. The top bar includes a back arrow, 'User', the app name 'DATA DIARY', and 'Name Logout' with a menu icon. The left sidebar shows a user profile for 'Name Administrator' with a 'New user' button and a plus icon. The main content area is titled 'Change administrator' and contains instructions: 'Please select the user to which the administration rights are to be transferred.' and 'Selected user should provide personal details in next step.' Below this is a 'Select user' dropdown menu. At the bottom are 'Cancel' and 'Next' buttons.

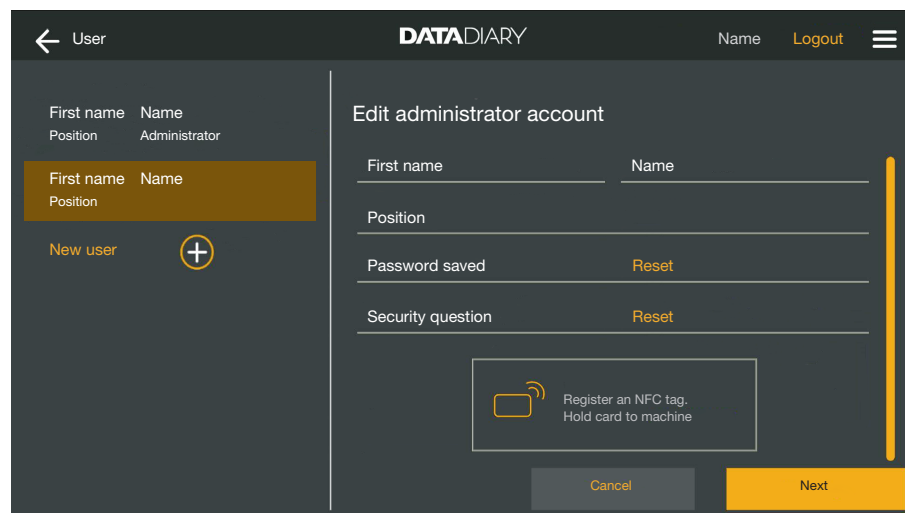
- Tap Select user and select a user from the list.
- Tap Next.

The screenshot shows the 'Edit administrator account' screen in the DataDiary app. The top bar is identical to the previous screen. The left sidebar is the same. The main content area is titled 'Edit administrator account' and contains form fields for 'First name', 'Name', 'Position', 'Password saved', and 'Security question'. Each field has a 'Reset' button next to it. Below the form is a section for 'Register an NFC tag. Hold card to machine' with an icon of an NFC tag. At the bottom are 'Cancel' and 'Next' buttons.

The view will then switch to the profile view for the new administrator.

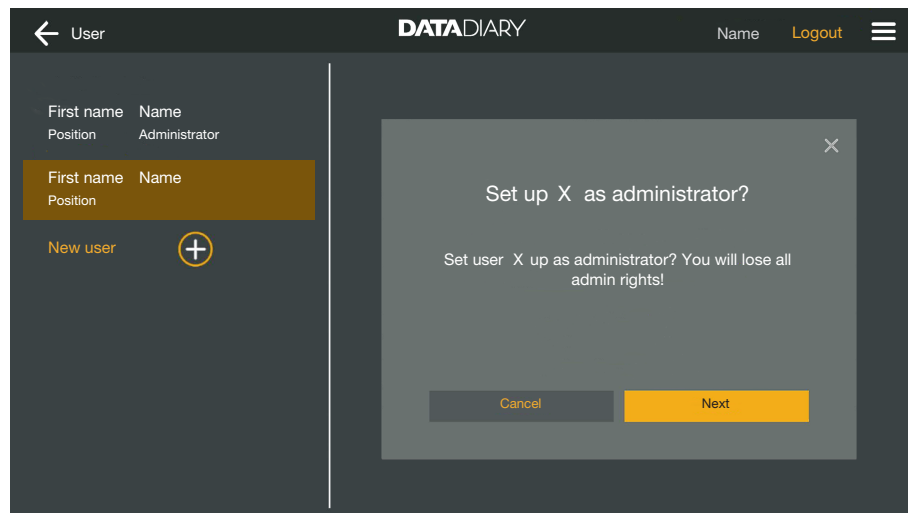
- The new administrator has to choose a new password and security question or reassign their NFC card. To do this, tap **Reset** in the respective fields.
- Tap **Next**.

A confirmation prompt will appear.

- Tap **Next** to transfer the administrator rights to the new administrator or **Cancel** to exit without saving.

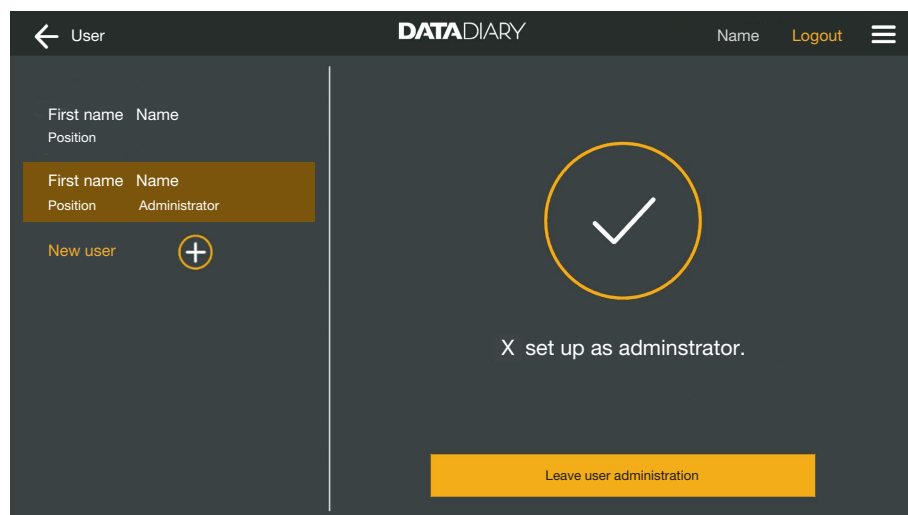If you choose to transfer the administrator rights to the new administrator, the following confirmation message will be displayed:

- Tap **Leave user administration** to return to the Dashboard.

Machines

Compatible machines

Not all machine types are compatible with the DataDiary app. Contact Miele to find out which machines you can connect to the app and which accessories you will need.

Optional components

To enable your Miele machine to use WiFi, it must be equipped with the right accessories. The accessories are available from Miele. Only use original Miele accessories. The accessories are supplied with their own sets of operating instructions.

- WiFi-enabled Miele XKM modules for cleaning machines
- Miele WiFi key (USB flash memory) for sterilisers

Connection options

You can connect the DataDiary app to your Miele machines via WiFi. Various connection options are available.

- 1:1 direct connection

The WiFi connection can be configured as a direct connection between 1 cleaning machine or steriliser and the DataDiary app. This is a 1:1 connection that is physically separate from other networks. This means that no other machines can be connected to the app and that the tablet will not be available for any other network activities, such as the Internet, while the connection is established.

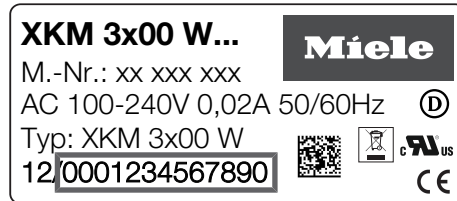
- Integration into an existing network

There is the option to integrate the DataDiary app as well as the cleaning machines and sterilisers into an existing network. In this case, multiple machines can be connected to the DataDiary at the same time. Depending on the network configuration, the tablet will then be available for other activities in the network.

If necessary, contact the Miele Customer Service Department for further information.
--

Adding a washer-disinfector

- Equip your cleaning machine with a WiFi-enabled Miele XKM module. Follow the installation instructions given in the module's operating instructions.
- Take a note of the module serial number. This is required as the password for establishing the WiFi connection. The serial number can be found on the data plate located on the side of the module.

The serial number of the module, without the initial zeroes, acts as the password. Example:

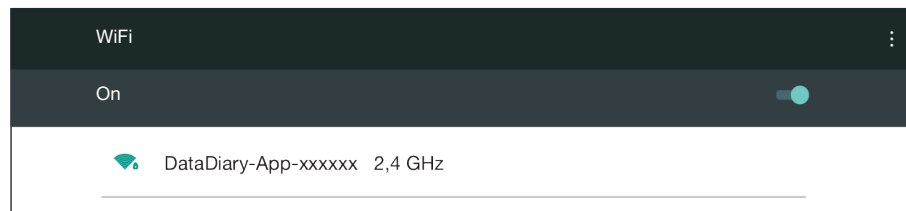
12/0001234567890

Password: **1234567890**

- Switch the cleaning machine on and configure the interface following the operating instructions for the machine and XKM module.
- Select the **Administration** option in the menu for the DataDiary app and log in as the app administrator.
- Tap the WiFi configuration option.

You will be taken to the control panel for your tablet.

- Open the settings to set up a WiFi interface and enable the WiFi.


- Select the right WiFi network and enter the password.
 - The WiFi SSID is DataDiary-AP-XXXXXX, where XXXXXX corresponds to the last 6 letters of the module MAC address.
 - The WiFi password is the serial number of the module without the initial zeroes.
- Check the WiFi settings.

Make sure that the WiFi is set to remain active when the tablet is in energy-saving mode (standby mode). This enables data to be transmitted between the machines and the app even in standby mode (see "Saving energy").


- Exit the control panel.
- Select the **Machines** option in the Administration menu.

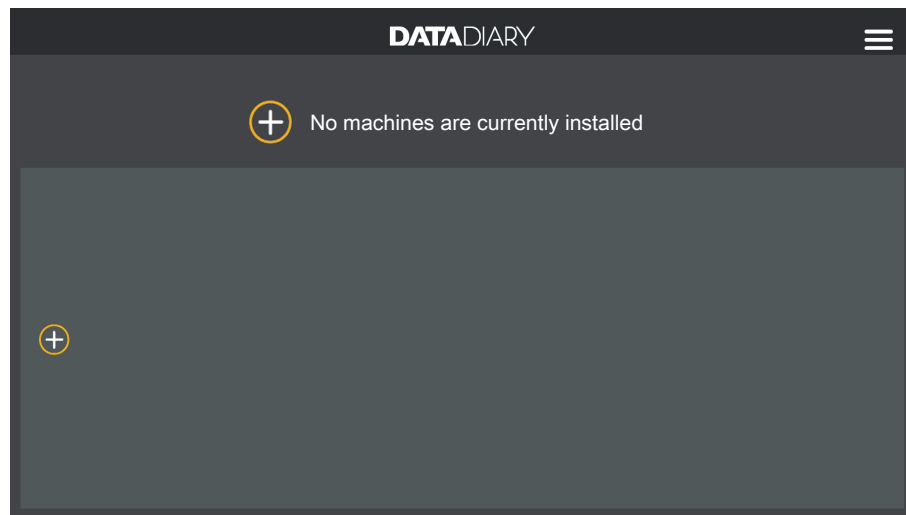Or:


- Tap the ⊕ icon on the Dashboard.

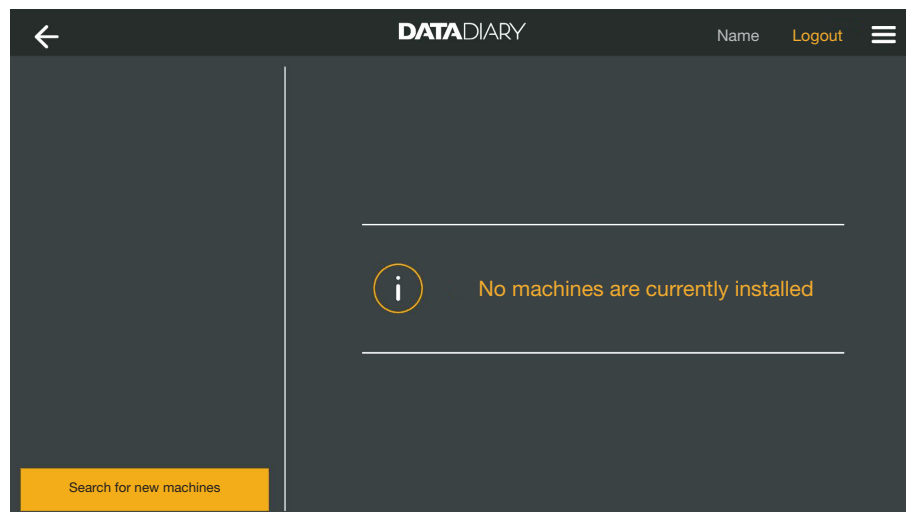The ⊕ icon is only displayed in the administrator view. If the ⊕ icon is not displayed in the administrator view, the maximum number of machines has already been registered with the app. It is not possible to add any more machines. Disconnect the connection to one or more machines as necessary.


- Tap Search for new machines.

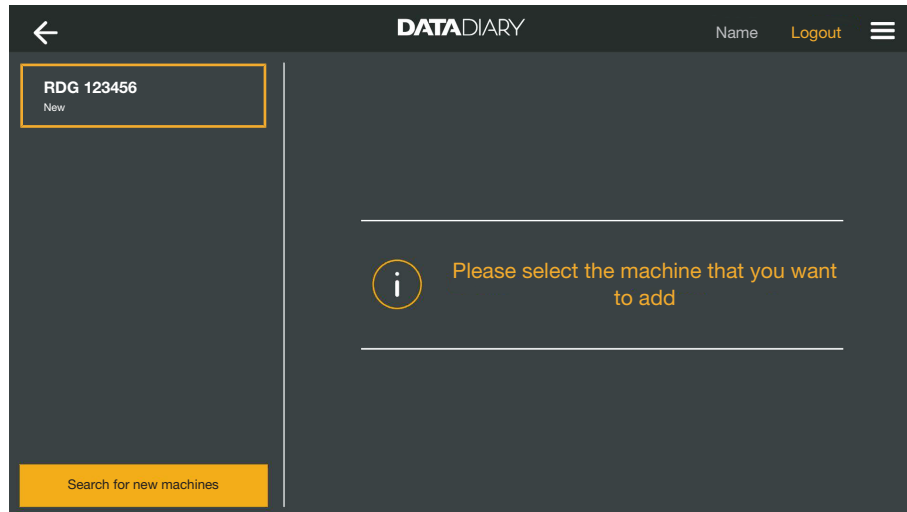The DataDiary app will then search for compatible machines. These will be listed in the left-hand column.

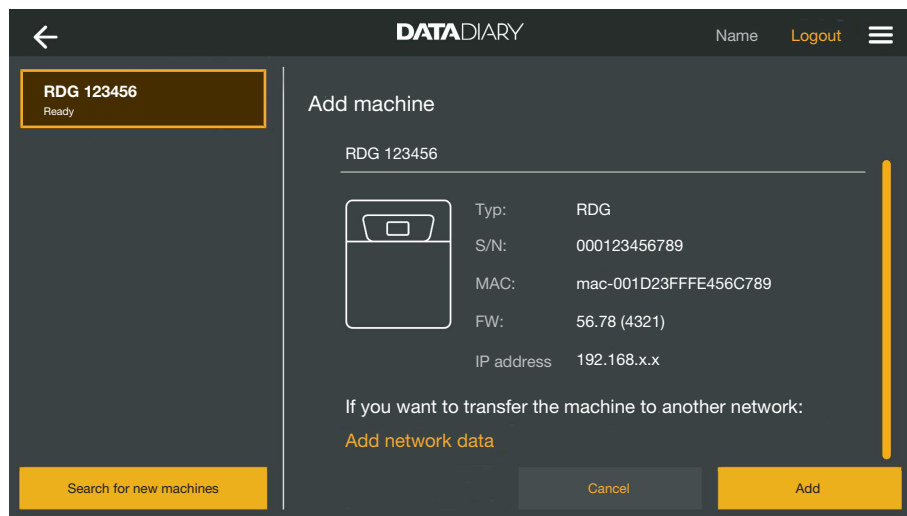- Tap the machine that you wish to connect to the app.

The data for the selected machine will be displayed in the right-hand column.

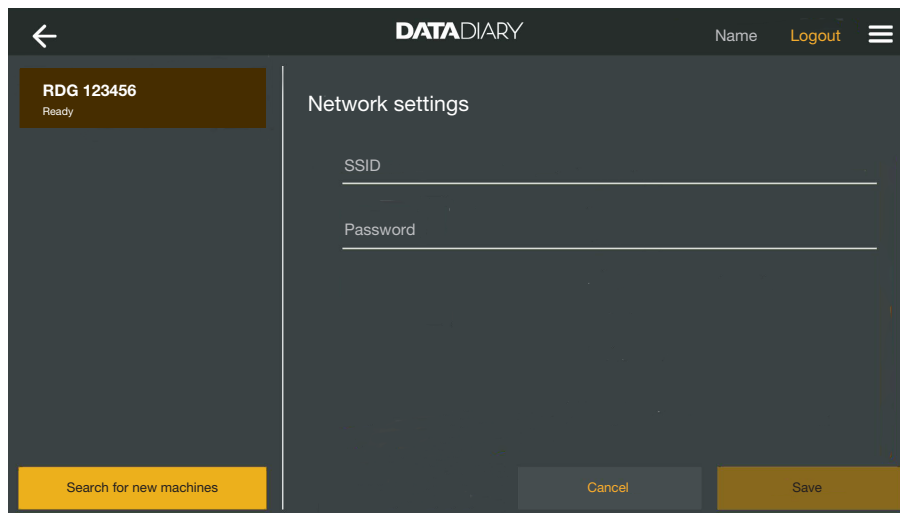
You now have 2 options for connecting the machine to the app:

- **1:1 direct connection:** tap **Add** to connect the machine to the app. If you establish a direct connection, it is not possible to add any more machines.

If necessary, you can change the name of the machine at this point simply by tapping the machine name and entering a new name of your choice.

The registration process is then complete.

- **Integration into an existing network:** in the right-hand column, tap **Add network data** underneath the machine data to register the cleaning machine in your network.


- Enter the SSID and the Password for the network (SSID: Service Set Identifier). When you want to complete a field, start by tapping it.
- Tap Save.

The cleaning machine is then registered in the network and the Data-Diary app automatically connects to the entered network.


If you switch WiFi network, the direct connection between the app and cleaning machine will be disconnected. You will then have to perform another machine search via the app to search the network for compatible machines.

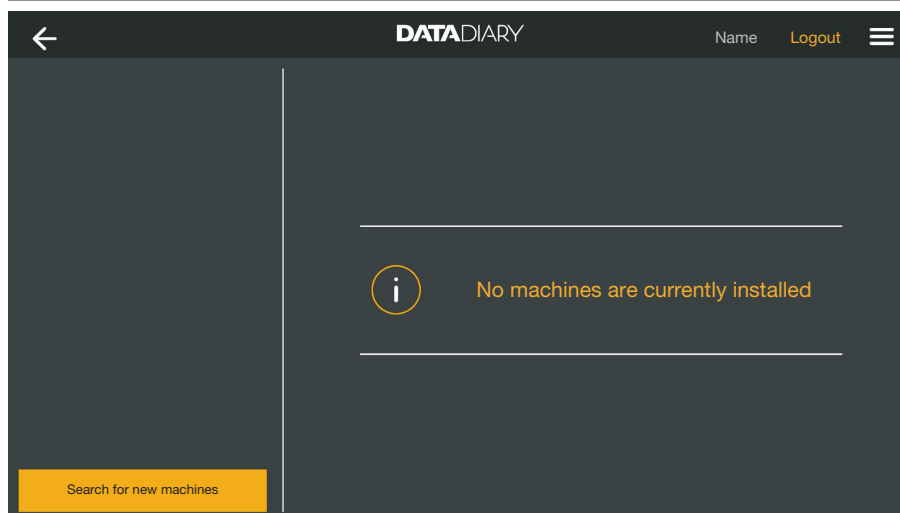Alternatively, you could choose to register additional cleaning machines with the network before starting the machine search. To do this, equip each of the cleaning machines with an XKM module and repeat the procedure for each individual machine.


- Tap Search for new machines.

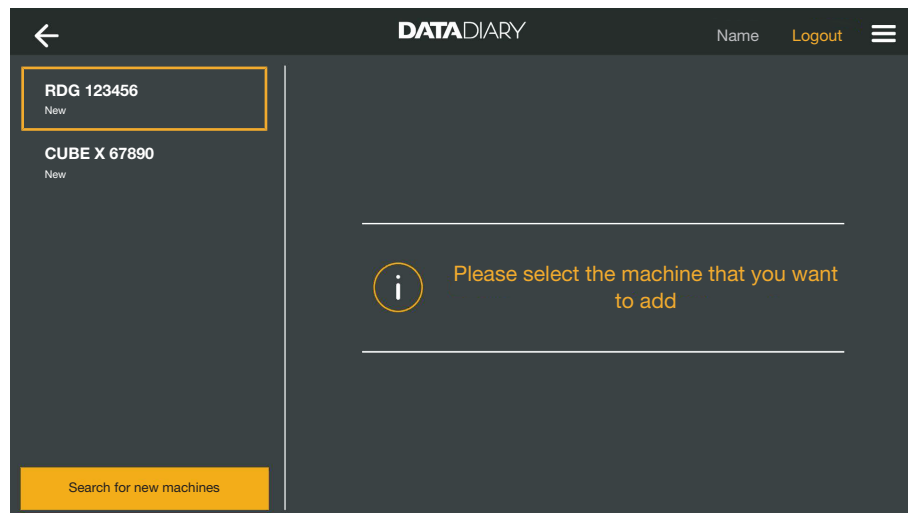The DataDiary app will then search for compatible machines. These will be listed in the left-hand column.

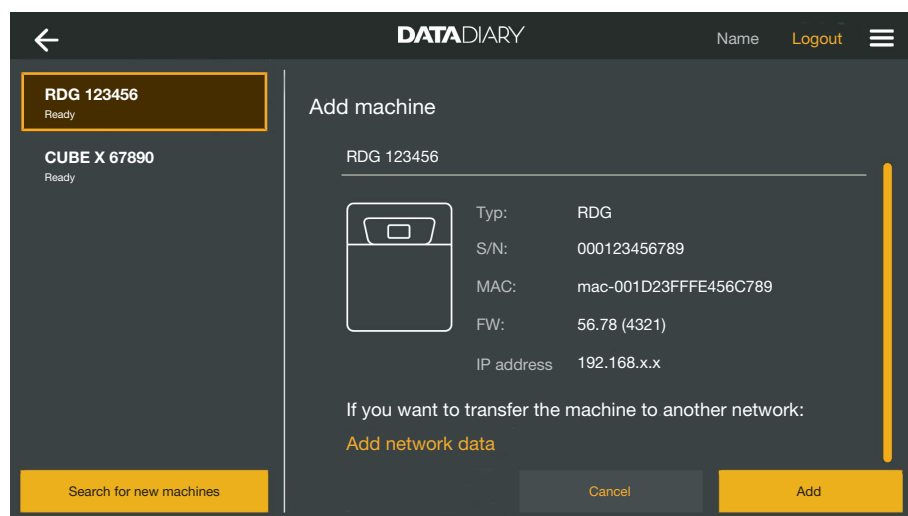- Tap the machine that you wish to connect to the app.

The data for the selected machine will be displayed in the right-hand column.

If necessary, you can change the name of the machine at this point simply by tapping the machine name and entering a new name of your choice.

- Tap Add to connect the machine to the app.

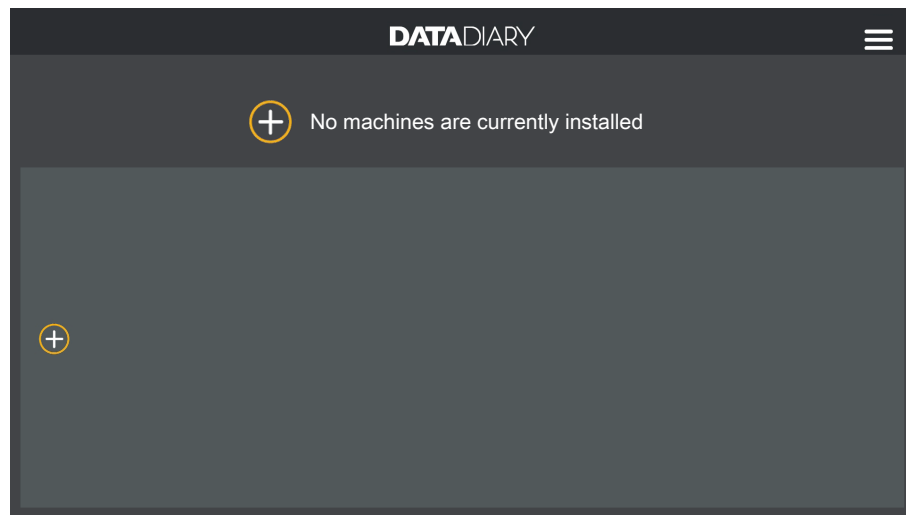
The connection to the app is now established. If necessary, you can also connect the additional machines shown in the left-hand column to the app.

en - Administration

Adding CUBE / CUBE X


- Equip the CUBE or CUBE X with a Miele WiFi key.
- Configure the WiFi interface via the CUBE/CUBE X display. To do this, consult the information in the operating instructions for the CUBE/CUBE X and the WiFi key.
- Select the **Administration** option in the menu for the DataDiary app and log in as the app administrator.
- Select the **Machines** option in the Administration menu.

Or:


- Tap the ⊕ icon on the Dashboard.

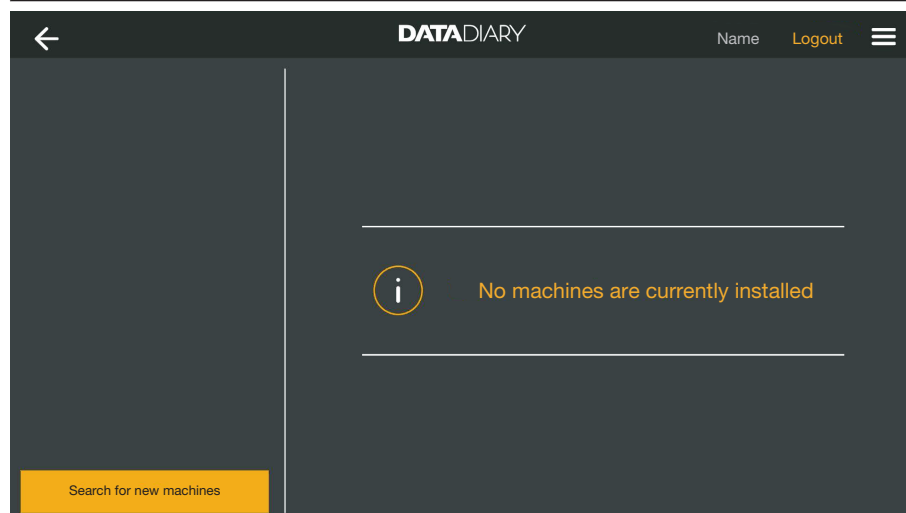The ⊕ icon is only displayed in the administrator view. If the ⊕ icon is not displayed in the administrator view, the maximum number of machines has already been registered with the app. It is not possible to add any more machines. Disconnect the connection to one or more machines as necessary.


- Tap Search for new machines.

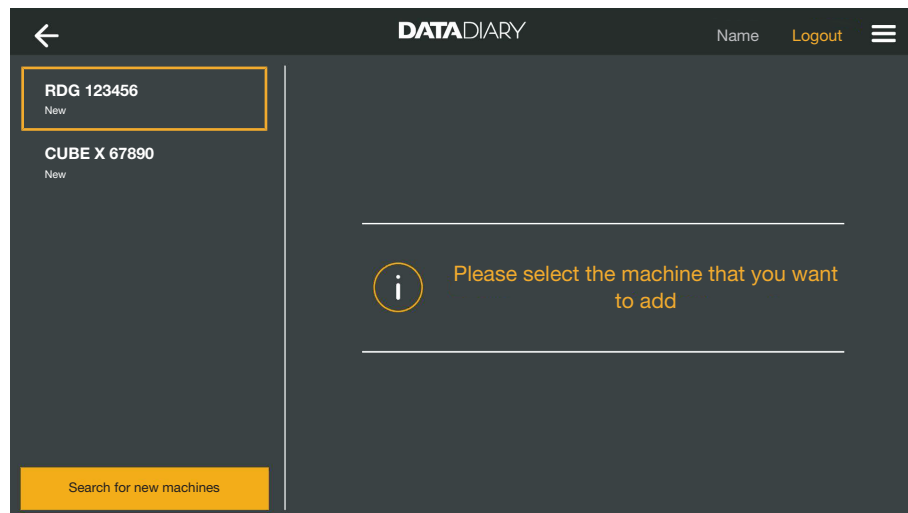The DataDiary app will then search for compatible machines. These will be listed in the left-hand column.

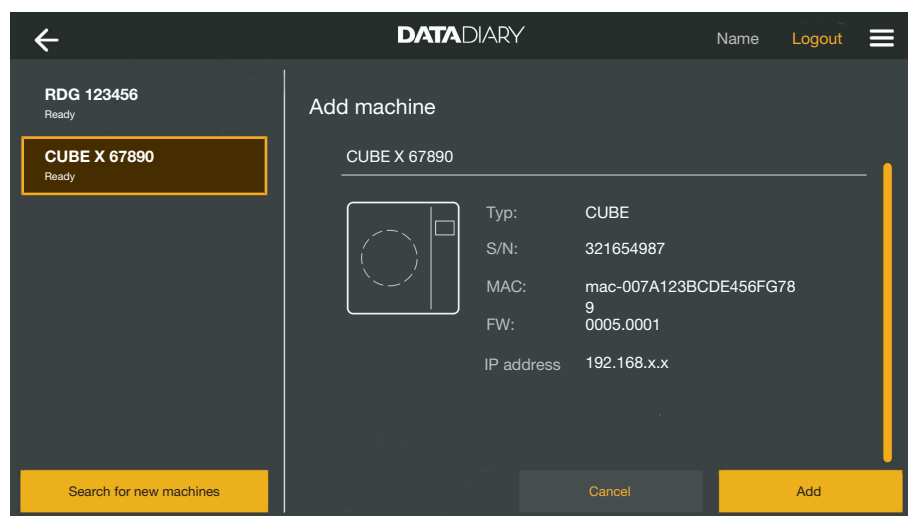- Tap the machine that you wish to connect to the app.

The data for the selected machine will be displayed in the right-hand column.

If necessary, you can change the name of the machine at this point simply by tapping the machine name and entering a new name of your choice.

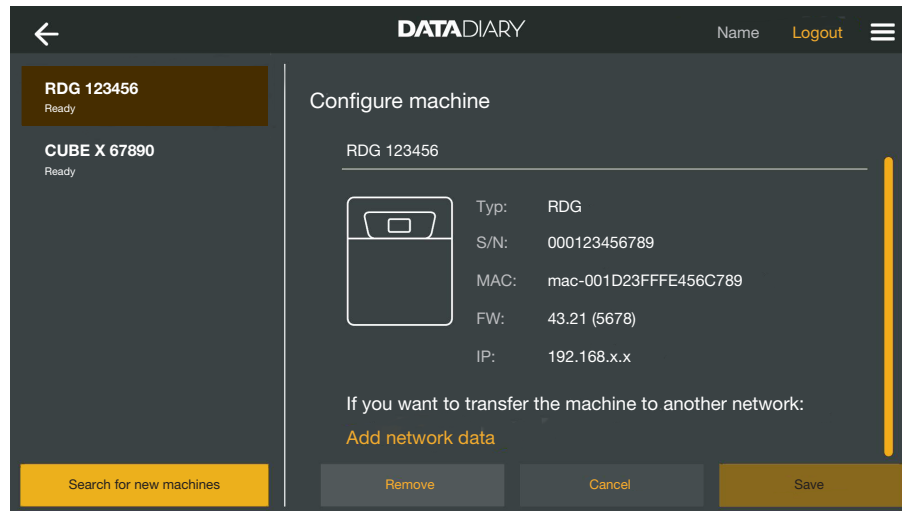
- Tap Add to connect the machine to the app.

The connection to the app is now established. If necessary, you can also connect the additional machines shown in the left-hand column to the app.

en - Administration

Machine management

- Select the Administration option in the menu for the DataDiary app and log in as the app administrator.
- Select the Machines option.


All machines that have been registered with the app are displayed in the left-hand column. The data for the selected machine is displayed in the right-hand column.

If necessary, you can change the name of the machine at this point simply by tapping the machine name and entering a new name of your choice.

You can then remove machines from the app or register a directly connected cleaning machine with your network.

Disconnecting the app connection

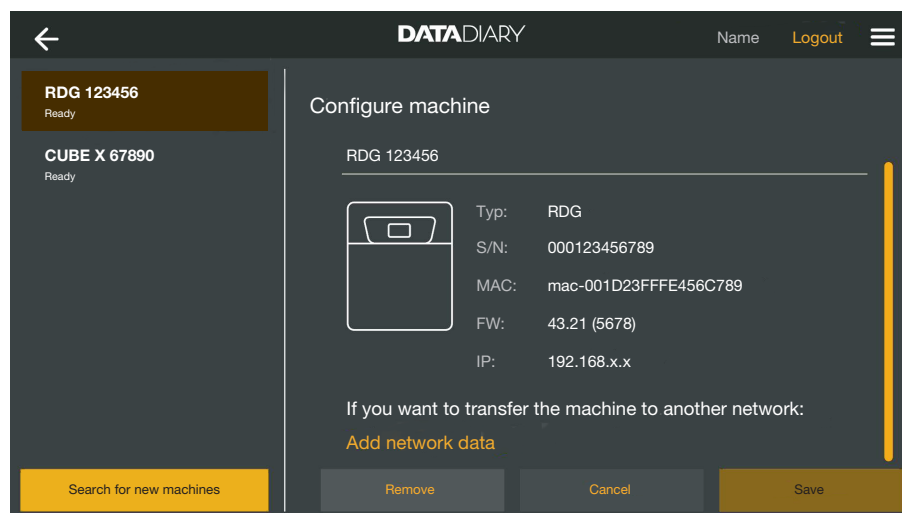- Tap the machine for which you wish to disconnect the app connection.


- Tap Remove.

The connection to the selected machine will be disconnected immediately. This action cannot be interrupted or undone. If you wish to reconnect the machine to the app, you will have to add the machine again.

Changing a direct connection to a network connection


For cleaning machines only:

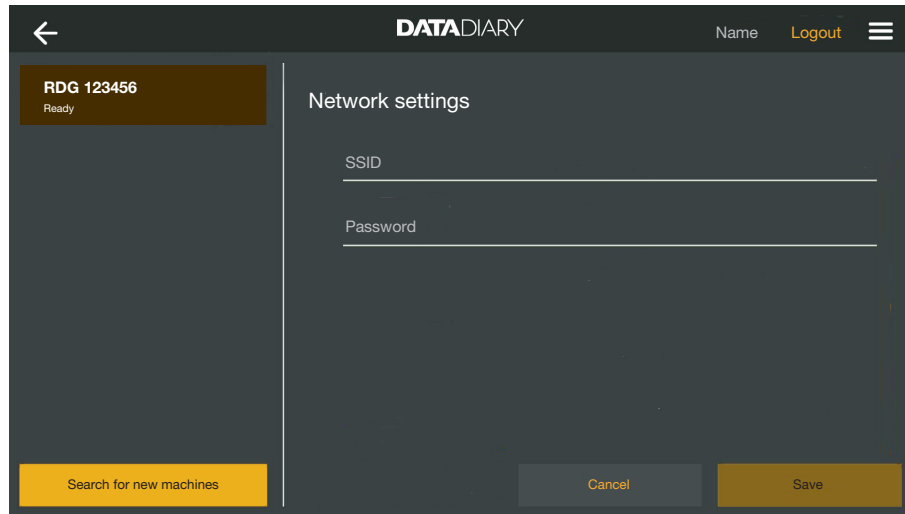- Select the cleaning machine and, in the right-hand column, tap Add network data underneath the machine data to register the cleaning machine in your network.


- Enter the SSID and the Password for the network (SSID: Service Set Identifier). When you want to complete a field, start by tapping it.
- Tap Save.

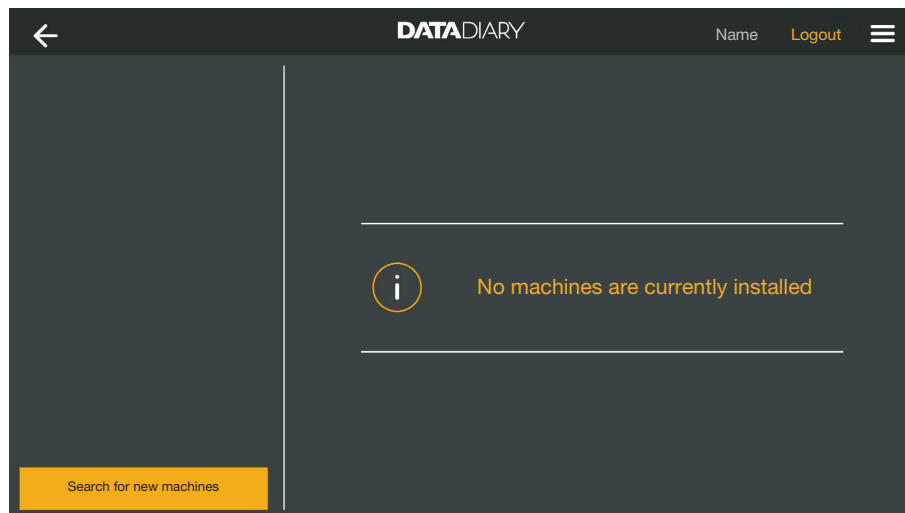The cleaning machine is then registered in the network and the DataDiary app automatically connects to the entered network.


- Tap Search for new machines.

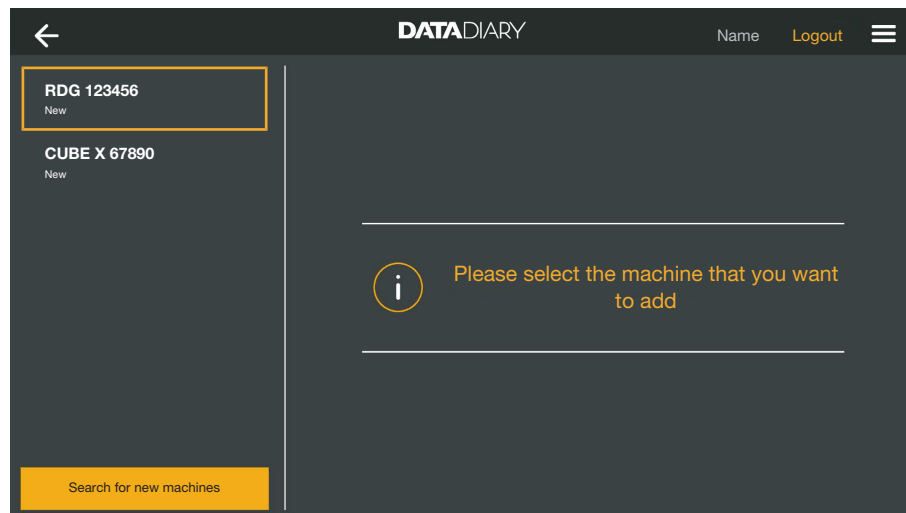The DataDiary app will then search for compatible machines. These will be listed in the left-hand column.

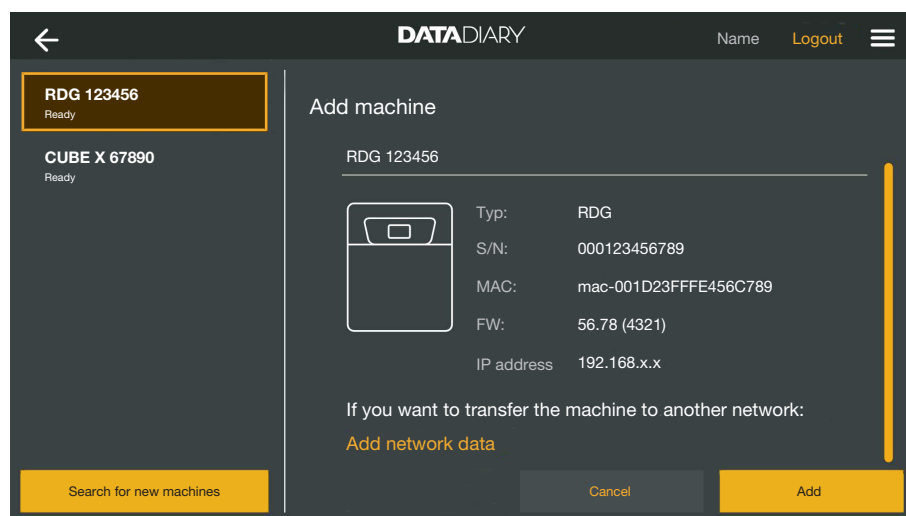- Tap the machine that you wish to connect to the app.

The data for the selected machine will be displayed in the right-hand column.

If necessary, you can change the name of the machine at this point simply by tapping the machine name and entering a new name of your choice.

- Tap Add to connect the machine to the app.

The connection to the app is now established. If necessary, you can also connect the additional machines shown in the left-hand column to the app.

WiFi configuration

 Unauthorised access poses a health risk. Settings for the cleaning machine, steriliser or app, e.g. parameters for disinfection or dispensing process chemicals, may be changed as a result of unauthorised access via the network. The cleaning machines, sterilisers and app should be operated in a separate network segment that is physically disconnected from other network segments, or access to the network should be restricted using a firewall or routers configured to provide protection against unauthorised access. Use strong passwords to protect access to the network. Limit access to the network to persons requiring access in the course of their work.

You will be taken to the control panel for your tablet. Here, you can check and change your WiFi settings.

Make sure that the WiFi is set to remain active when the tablet is in energy-saving mode (standby mode). This enables data to be transmitted between the machines and the app even in standby mode (see “Saving energy”).

- Select the **Administration** option in the menu for the DataDiary app and log in as the app administrator.
- Tap the **WiFi configuration** option.

You will be taken to the control panel for your tablet.

- Check and change the WiFi settings if necessary. Exit the control panel if you wish to return to the app.

Routers

Routers with access restrictions

Routers with access restrictions only allow certain devices to access the network, e.g. with the aid of a MAC filter.

- Please contact your system administrator so they can set up the relevant access.

Routers with time restrictions

Routers with time restrictions allow data to be transmitted within a particular time frame.

- If the process ends outside of this time frame, the process data must be downloaded manually on the next working day.

Range and availability of WiFi signal

The WiFi range or signal strength is dependent on the structural environment. Reinforced concrete walls and ceilings, metal surfaces, metal-vaporised films, glass screens and fire protection constructions can muffle or divert the signal, therefore reducing the range.

For example, if the cleaning machines or sterilisers are located under a metal worktop or surrounded by metal cabinets, the signal will be almost completely blocked.

Connection problems between the app and the machines can be caused by many factors. Check the points listed below, for example.

Also observe the instructions given in the operating instructions for your Miele machines and accessories as well as for the components of your local network. Consult your network administrator if necessary.

Archiving reports

Save location

A PDF file is automatically generated from any report that is edited, released or cancelled using the DataDiary app. The PDF files are stored in the tablet's document memory. The save location varies depending on the tablet's operating system.

Alternatively, the reports can also be backed up to SD cards.

 The tablet's internal memory and the SD cards are not a suitable location for permanently archiving reports.

For this reason, you should regularly transfer the reports to a storage medium with a backup solution.

Saving reports to SD cards

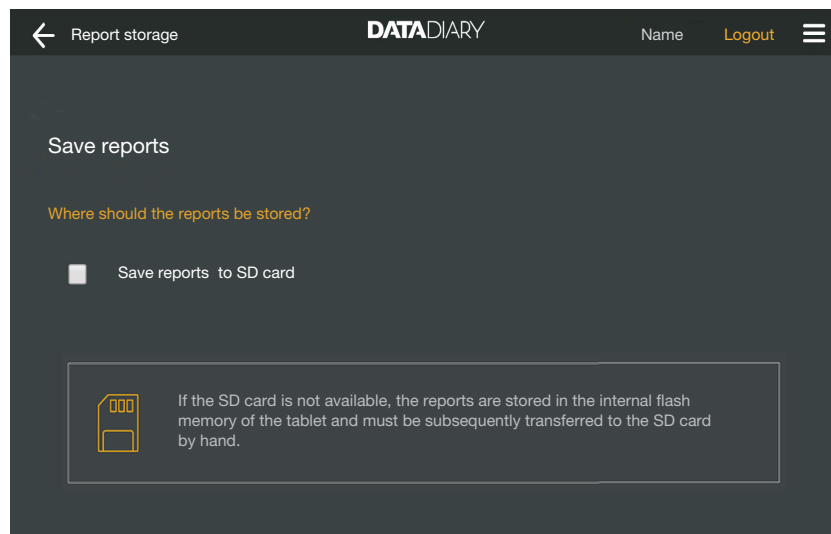
- Select the Administration option in the menu for the DataDiary app and log in as the app administrator.

To save reports to SD cards, the tablet must either have an SD card slot or be connected to an SD card reader.

- Insert a suitable SD card into the slot in your tablet or into the card reader.

Make sure that the card's write protection is disabled. The switch for the write protection must be pointing towards the card's contacts or moved so that it is.

- Select the Report storage option.

- Tick the Save reports to SD card checkbox.
 - If the checkbox is ticked, the SD card is defined as the save location for the reports and a corresponding folder will be set up on the card. The path is predefined and cannot be changed.
 - If the checkbox is not ticked, the reports are stored in the tablet's document memory.

If no SD card is available or if write protection is enabled, the reports are temporarily stored in the tablet's document memory. They must then be manually transferred to the SD card. The reports will not be automatically transferred after a writable card has been inserted.

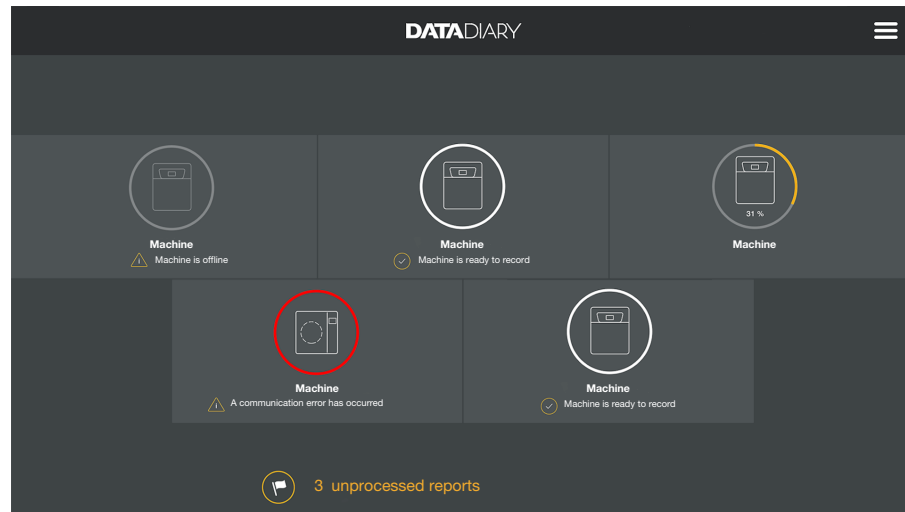
- Restart the app.

Live report


Washer-disinfectors transmit the cycle reports to the DataDiary app while the process is in progress. This enables the reports to be tracked live on the tablet.

The CUBE and CUBE X sterilisers transmit the reports as a group at the end of a process, as soon as the door is unlocked.

You can view the last report for each machine.

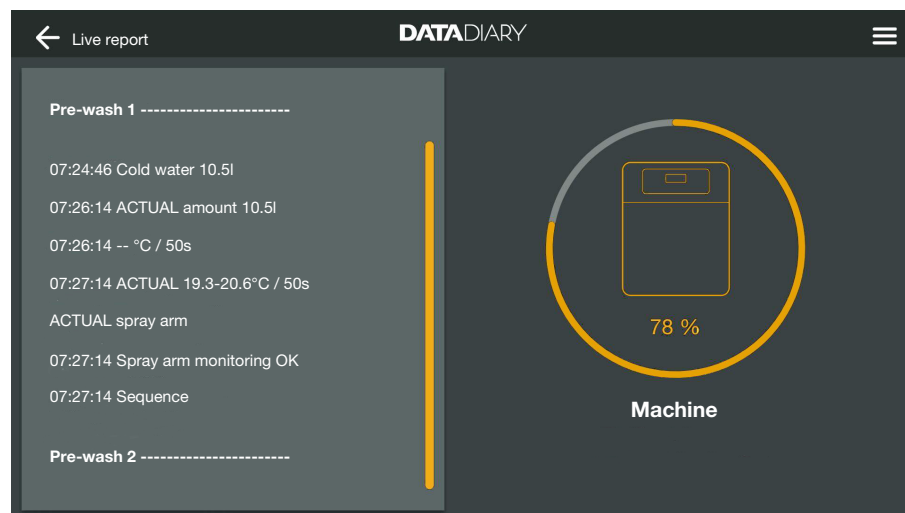- To do this, tap a machine on the Dashboard.


The live report is then displayed:

- The left-hand column shows the individual process steps in chronologically descending order.
- The right-hand column displays the name of the machine. If a process is ongoing for a washer-disinfector, the progress is also shown in % (per cent).

It is not possible to make entries in this view.

Editing reports

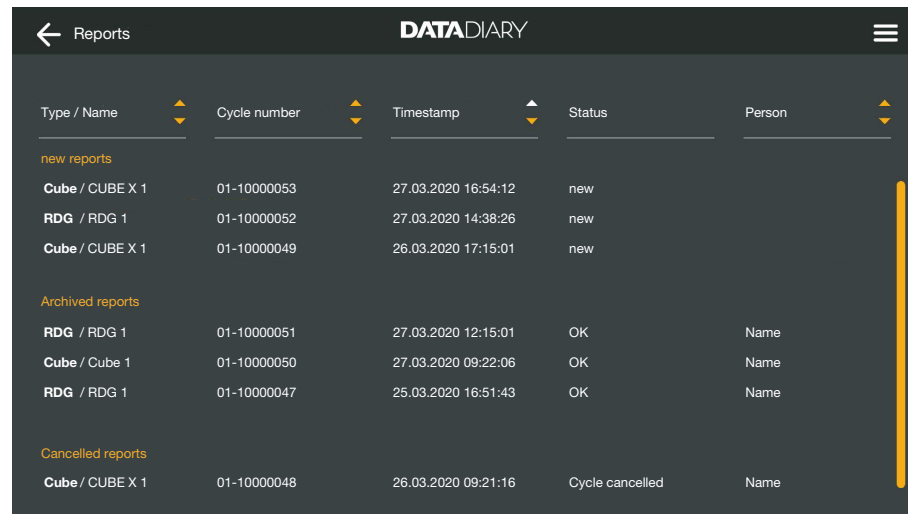
- Retrieving reports** ■ Open the menu for the DataDiary app and select the Reports option.

Type / Name	Cycle number	Timestamp	Status	Person
new reports				
Cube / CUBE X 1	01-1000053	27.03.2020 16:54:12	new	
RDG / RDG 1	01-1000052	27.03.2020 14:38:26	new	
Cube / CUBE X 1	01-1000049	26.03.2020 17:15:01	new	
Archived reports				
RDG / RDG 1	01-1000051	27.03.2020 12:15:01	OK	Name
Cube / Cube 1	01-1000050	27.03.2020 09:22:06	OK	Name
RDG / RDG 1	01-1000047	25.03.2020 16:51:43	OK	Name
Cancelled reports				
Cube / CUBE X 1	01-1000048	26.03.2020 09:21:16	Cycle cancelled	Name

The reports are listed on the display. Unedited reports appear at the top in a dedicated category. You can use the ▲ and ▼ arrows to sort the view. The type of sorting that is currently being used is indicated by a white arrow.

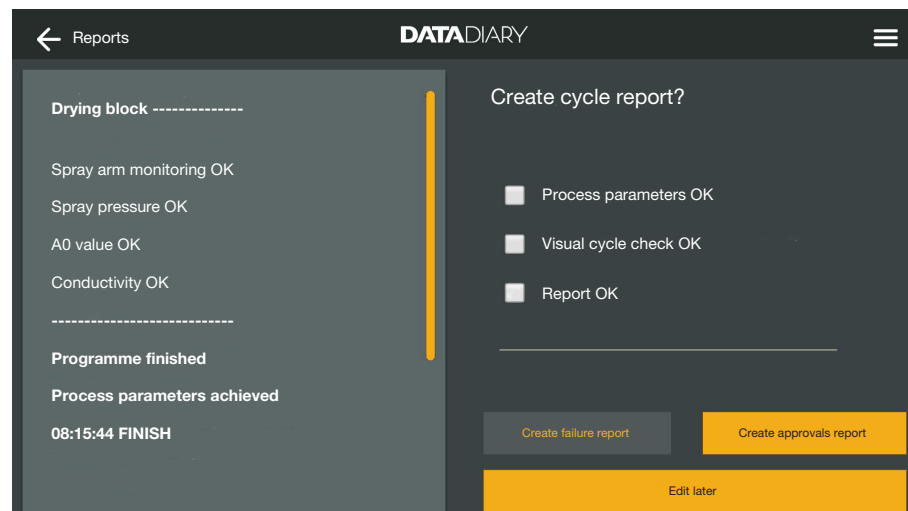- Checking reports** ■ Select one of the incomplete reports from the list.

Drying block -----

Spray arm monitoring OK

Spray pressure OK

A0 value OK

Conductivity OK

Programme finished

Process parameters achieved

08:15:44 FINISH

Create cycle report?

Process parameters OK

Visual cycle check OK

Report OK

Create failure report Create approvals report

Edit later

The screen for editing the reports is divided into two:

- The left-hand side shows the individual process steps in chronologically descending order.
- The right-hand side lists the editing options.

en - Reports

Checking process parameters

At the end of a reprocessing programme or sterilisation cycle, a message is issued on the machine display to indicate whether the process was completed successfully or not, e.g.:

- Process parameters OK
The process was finished without any problems.
- Process parameters not achieved
During the process, problems occurred which meant that the desired process result could not be achieved or which resulted in the process being interrupted. This message usually appears together with one or more fault codes/messages.

A corresponding message is also documented in the cycle report and can be tracked in the report views of the DataDiary app.

- In the left-hand column, scroll down to the end of the report.
- Check whether the message from the report matches the message on the machine.

Check the reports as soon as possible after the processes have finished to compare the messages on the machine displays with the messages on the reports.

- Process parameters OK **message:**
In the right-hand column, tick Process parameters OK.
- Process parameters not achieved **message:**
Do not tick Process parameters OK in this case. Check the report for fault messages and cancel it if necessary (see “Cancelling reports”).

Visual check

- Check the process result by carrying out a visual inspection, for example.
- The process result has been achieved:
In the right-hand column, tick the Visual cycle check OK option.
- There is a problem with the process result:
Do not tick Visual cycle check OK in this case. Check whether the cycle needs to be completed again and the report needs to be cancelled (see “Cancelling reports”).

Checking for discrepancies

- Finally, check the report one more time in case there are any discrepancies – for example, the time being set to winter even though summer now applies.

The dates and times indicated in the reports are taken from the cleaning machines and sterilisers. If the dates or times are wrong, they must be reset on the machines. To do this, follow the operating instructions for the machines. The tablet's date and time is only consulted when documenting report editing.

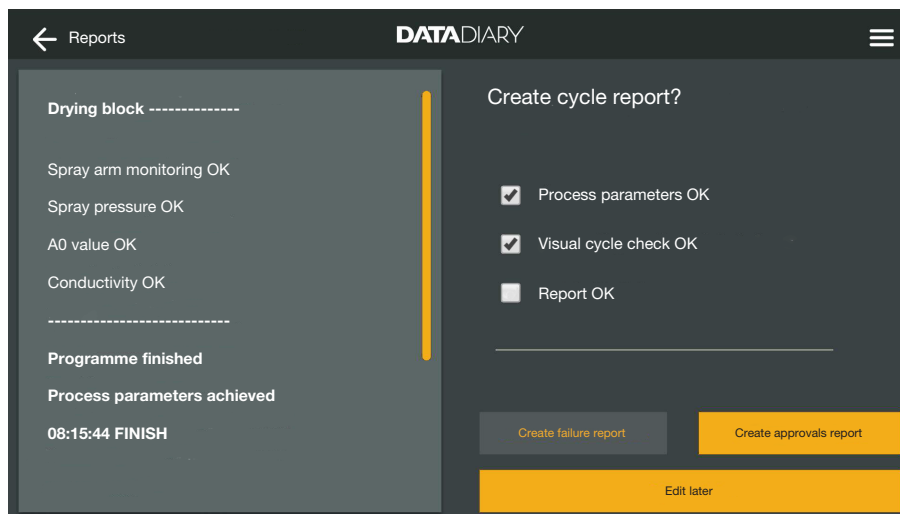
- If the report is correct, tick the Report OK checkbox.
- If you do find any discrepancies, such as an incorrect time, you can still approve the report by ticking the Report OK checkbox. Document the discrepancy in the free text area provided below.

Aborting editing

Select **Edit later** to abort the process of editing the report. This will clear any ticks you have placed in boxes and anything you have entered in the comment area.


Approving reports

Approvals are final and cannot be undone. It is not possible to make changes later on.
If you are unsure, cancel the process.

If the Process parameters OK and Visual cycle check OK checkboxes are ticked, the Create approvals report button is enabled.

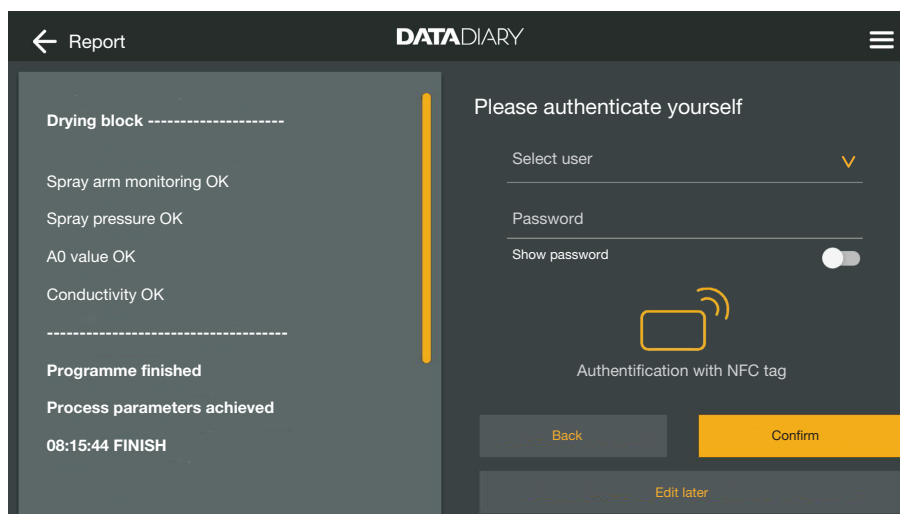- If you need to, you can enter a comment in the free text area under the checkboxes.
- Press the Create approvals report button to approve the report.


You will then be prompted to authenticate yourself.


- Log in with your user name and password.

Or:

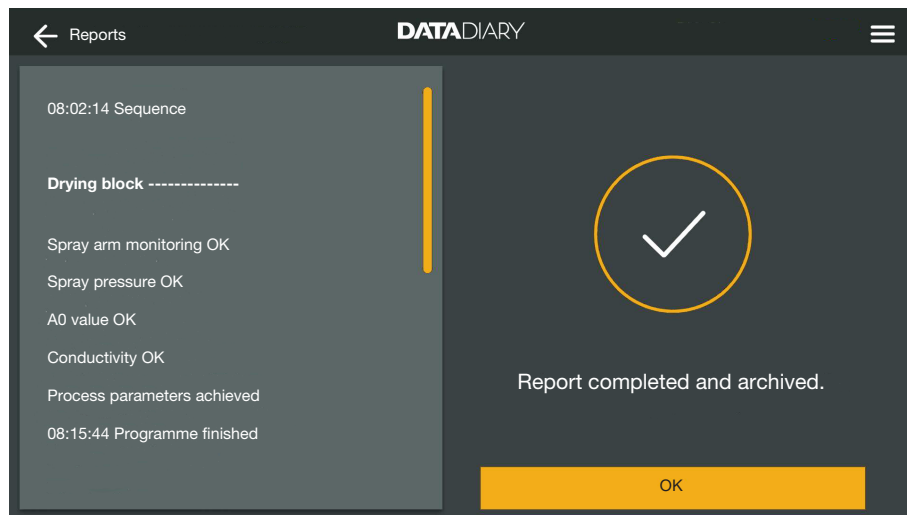- Log in using your NFC card by holding it against the NFC sensor in your tablet. The sensor is usually located on the back of the tablet.

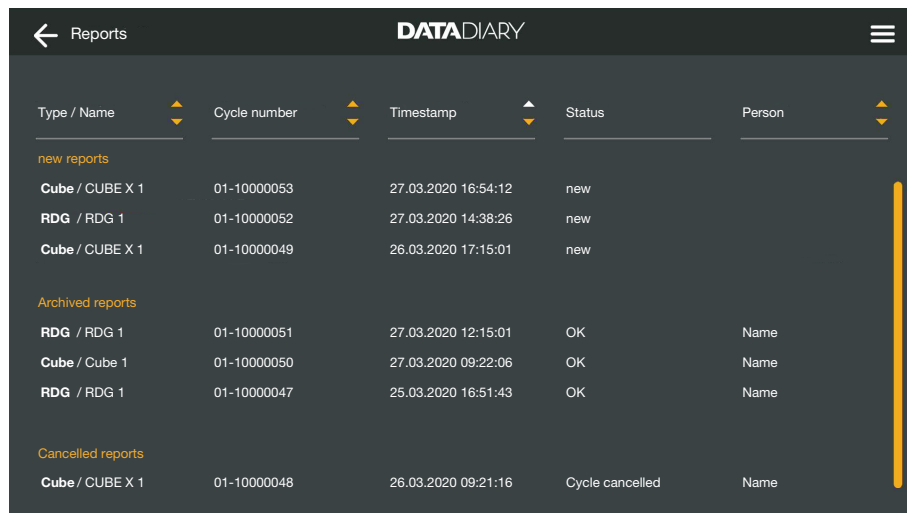
Once you have correctly authenticated yourself, a PDF will be generated from the report and stored (see “Archiving reports”).


The report will be stored in the list of edited reports and the logged-in user will be stored as the editor (Person).

Cancelling reports

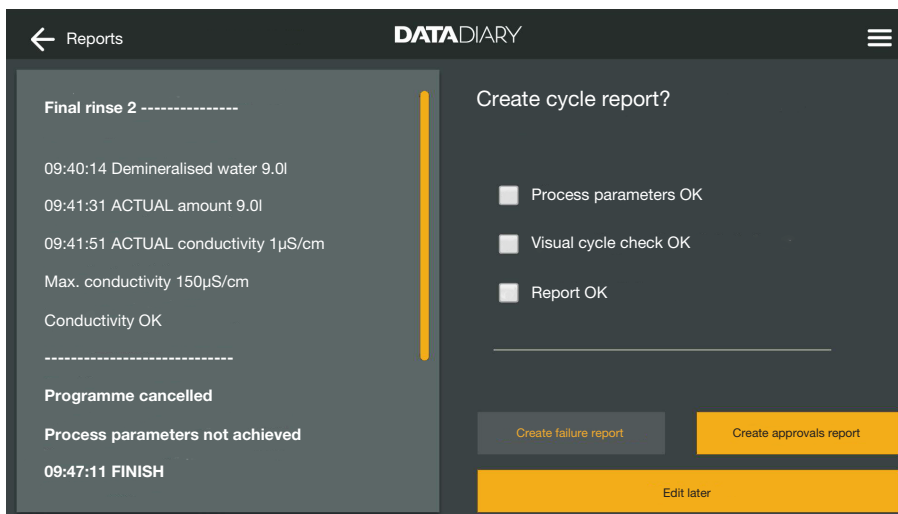Cancellations are final and cannot be undone. It is not possible to make changes later on.
If you are unsure, cancel the process.

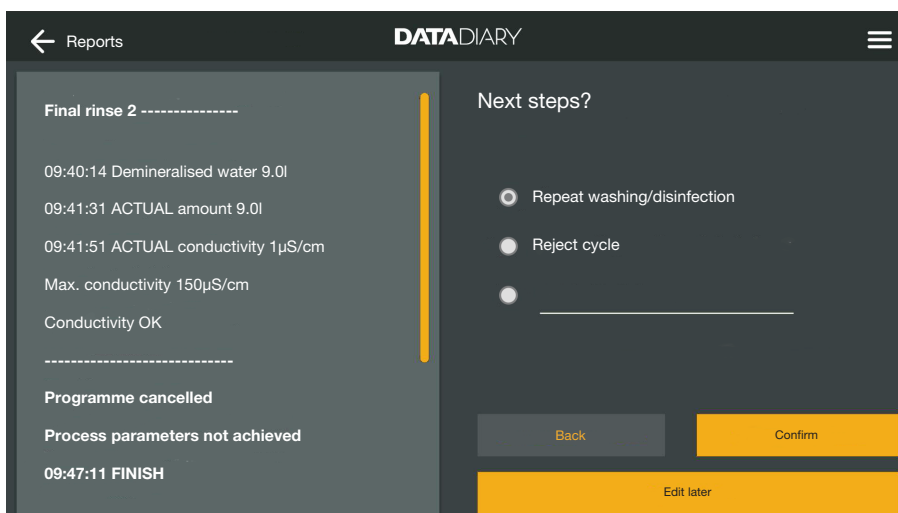
- If the report is not OK, tap the **Create failure report** button.

You must specify how to proceed with the cycle. You can either choose one of the suggestions, or choose the third option and enter free text.

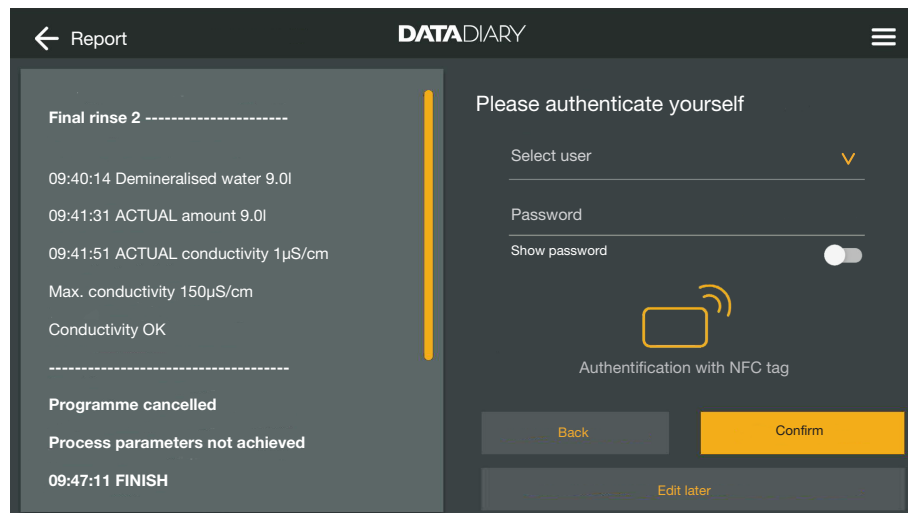- Choose an option and tap the **Confirm** button.

You will then be prompted to authenticate yourself.

- Log in with your user name and password.

Or:

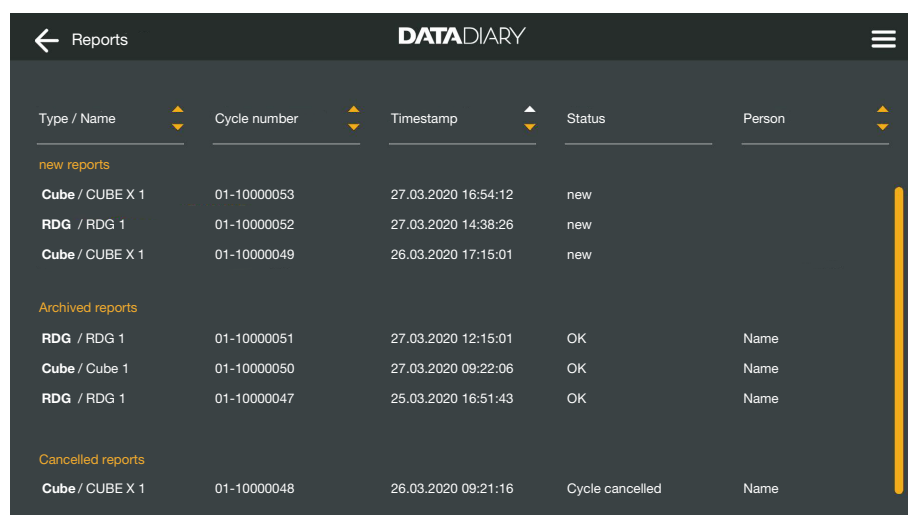
- Log in using your NFC card by holding it against the NFC sensor in your tablet. The sensor is usually located on the back of the tablet.

The report will be stored in the list of edited reports and the logged-in user will be stored as the editor (Person).


Viewing reports

Provided the edited reports have been stored in the tablet's internal memory or on the SD cards, you can view them on the tablet.

- Open the menu for the DataDiary app and select the Reports option.

- Tap one of the edited reports.

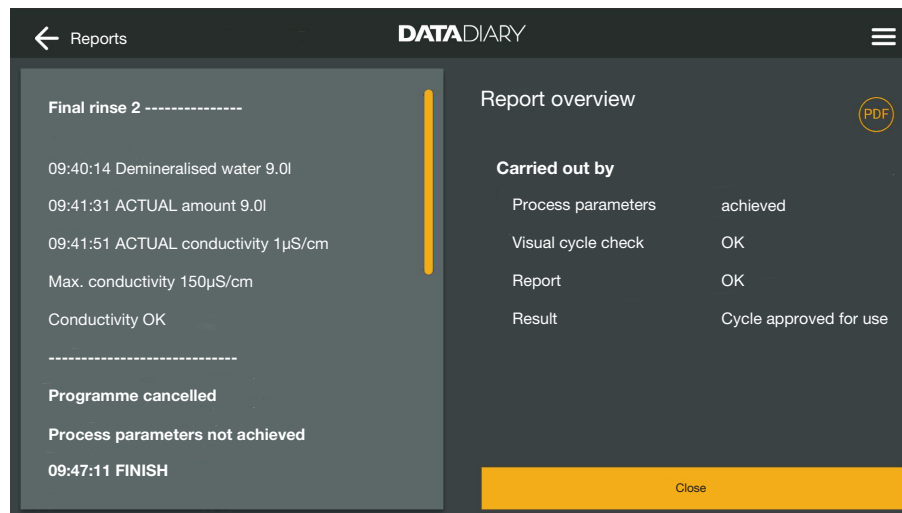The screen splits into two:

- The left-hand side shows the individual process steps in chronologically descending order.
- The right-hand side displays an overview of which user has edited the report and what the result was.
- You can view the PDF version of the report by tapping the circled PDF icon.
- Tap Close to return to the report overview.

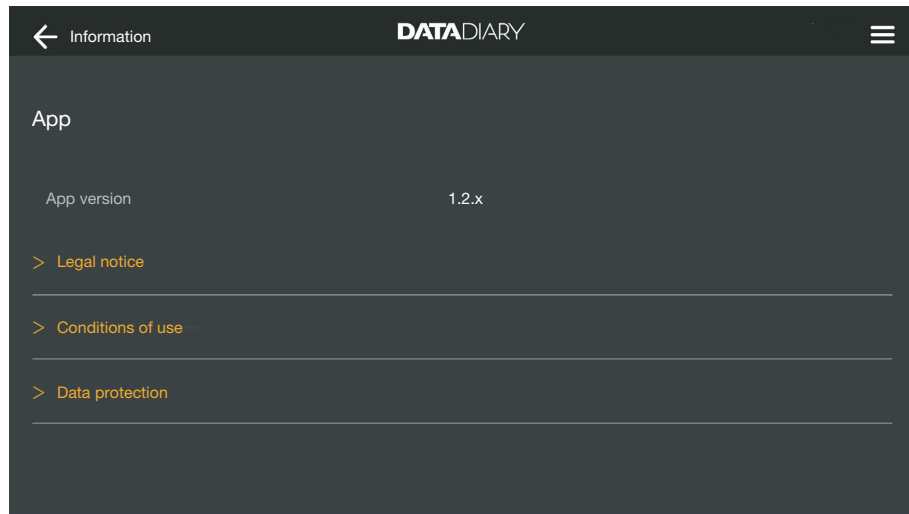
Retrospective output of reports

The reports are stored in the memory of the cleaning machines or sterilisers for a certain period of time. If there are WiFi problems during data transmission or the tablet's battery dies, the cycle reports can be transmitted to the DataDiary app retrospectively. More details are provided in the respective operating instructions for the machines.

Information

Here, you can view information about the app.

- Open the menu for the DataDiary app and select the Information option.

The following information about the DataDiary app is provided:

- DataDiary app version (App version)
 - Legal notice (Legal notice)
 - Conditions of use (Conditions of use)
 - Information regarding data protection (Data protection)
- You can view the legal notice, conditions of use and information regarding data protection by tapping the respective field.
 - Exit the menu using the arrow icon ← at the top left.

en - Problem solving guide

The following guide should help you to find the reason for a fault and to correct it. Also observe the instructions given in the operating instructions for your Miele machines and accessories as well as for the components of your local network. Consult your network administrator if necessary.

Problem	Cause and remedy
Reports are not transmitted	The tablet's energy-saving mode has interrupted the connection between the app and machines. <ul style="list-style-type: none"> ■ Configure your tablet's energy-saving mode as described in "Saving energy".
	The app was not yet ready for operation. <ul style="list-style-type: none"> ■ Make sure you start the DataDiary app before you start a process on a machine.
	Automatic shutdowns at defined times may be set for the tablet or router. <ul style="list-style-type: none"> ■ Make sure you start the processes in enough time for them to finish before the automatic shutdown or adjust the times for the automatic shutdown.
	Reports are kept in the machine's memory for a certain period of time and can still be retrieved during this time. To do this, consult the information in operating instructions for the machines.
Reports that have already been edited are no longer displayed	Once the reports have been transferred to a suitable storage medium for permanent archiving, they can generally no longer be viewed via the app.
Insufficient WiFi signal	The WiFi signal is too weak or switched off. <ul style="list-style-type: none"> ■ Check whether WiFi is enabled on the tablet and whether the WiFi accessories on the machines are ready for operation. ■ Check whether the tablet and the machines are within range of the WiFi signal.
	The range and availability of the WiFi signal are influenced by various factors, see "WiFi configuration".
Machines not displayed on the Dashboard	Only 1 machine is displayed on the Dashboard. <ul style="list-style-type: none"> ■ Check the WiFi connection. A direct connection may have been set up, see "Adding a washer-disinfector". New washer-disinfectors are connected to the app as a direct connection. To establish a network connection, you have to be signed into the local network via the app.

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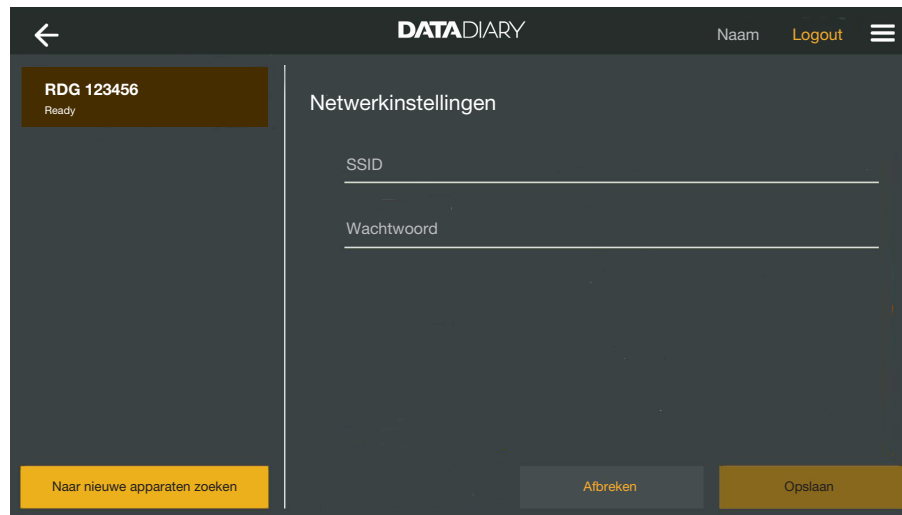
The screenshot shows the 'Netwerkinstellingen' (Network Settings) screen in the DataDiary app. At the top left, there is a back arrow and the app name 'DATA DIARY'. At the top right, there are 'Naam' and 'Logout' options, and a menu icon. On the left side, a card displays 'RDG 123456' and 'Ready'. The main area is titled 'Netwerkinstellingen' and contains two input fields: 'SSID' and 'Wachtwoord'. At the bottom, there are three buttons: 'Naar nieuwe apparaten zoeken' (left), 'Afbreken' (middle), and 'Opslaan' (right).

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The screenshot shows the 'Protocollen' screen in the DATA DIARY application. On the left, a list of steps is shown: 'Drying block', 'Spray arm monitoring OK', 'Spray pressure OK', 'A0 value OK', 'Conductivity OK', 'Programme finished', and 'Process parameters achieved' with a timestamp '08:15:44 FINISH'. On the right, a question asks 'Kan het chargeprotocol worden opgesteld?' with three checkboxes: 'Aan procesparameters voldaan' (checked), 'Visuele chargecontrole OK' (checked), and 'Protocol in orde' (unchecked). At the bottom, there are three buttons: 'Storeringsprotocol opstellen' (disabled), 'Vrijgaveprotocol opstellen' (active), and 'Later bewerken'.

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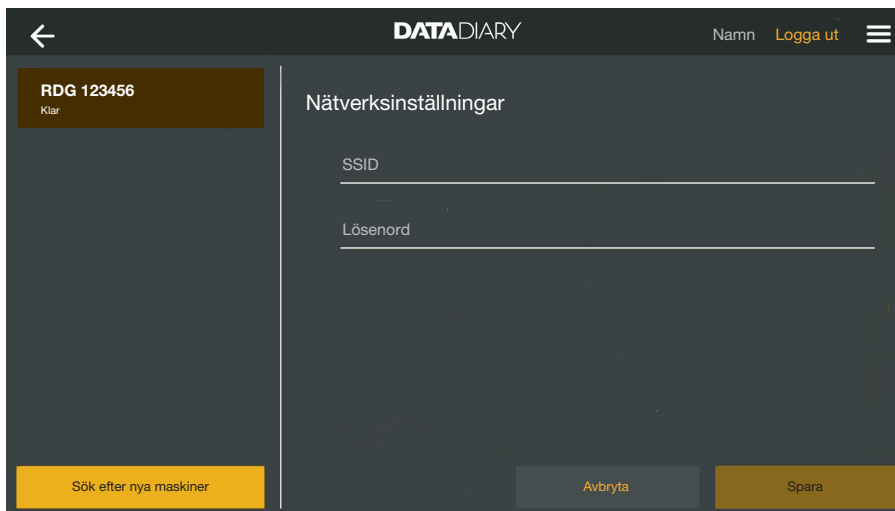
The screenshot shows the 'Nätverksinställningar' (Network Settings) screen in the DataDiary app. The top bar includes a back arrow, the 'DATA DIARY' logo, and 'Namn Logga ut' with a menu icon. On the left, a device card for 'RDG 123456' is shown with a 'Klar' status. The main area contains two input fields: 'SSID' and 'Lösenord'. At the bottom, there are three buttons: 'Sök efter nya maskiner' (yellow), 'Avbryta' (grey), and 'Spara' (yellow).

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The screenshot shows the 'Protokoll' screen in the DATA DIARY application. On the left, a log for 'Final rinse 2' is displayed with the following details: 09:40:14 Demineralised water 9.0l, 09:41:31 ACTUAL amount 9.0l, 09:41:51 ACTUAL conductivity 1µS/cm, Max. conductivity 150µS/cm, and Conductivity OK. Below the log, it states 'Programme cancelled' and 'Process parameters not achieved', followed by '09:47:11 FINISH'. On the right, a dialog box asks 'Kan batchprotokollet skapas?' (Can the batch protocol be created?). It contains three checkboxes: 'Processparameter uppfylld' (unchecked), 'Visuell batchkontroll OK' (unchecked), and 'Protokoll OK' (unchecked). At the bottom of the dialog, there are three buttons: 'Skapa annulleringsprotokoll' (grey), 'Skapa frigivningsprotokoll' (yellow), and 'Bearbeta senare' (yellow).

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This screenshot is similar to the previous one, showing the same log for 'Final rinse 2'. The dialog box on the right now asks 'Hur vill du gå vidare med batchen?' (How do you want to proceed with the batch?). It features three radio button options: 'Ny rengöring och desinfektion' (selected), 'Sortera ut batchen' (unselected), and an empty radio button with a text input field below it. At the bottom of the dialog, there are three buttons: 'Tillbaka' (grey), 'Bekräfta' (yellow), and 'Bearbeta senare' (yellow).

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- Ansvarig utgivare (Miele AB)